


1. Login as Administrator (admin: admin)



2. Press on License Activation




3. Choose I have a license key

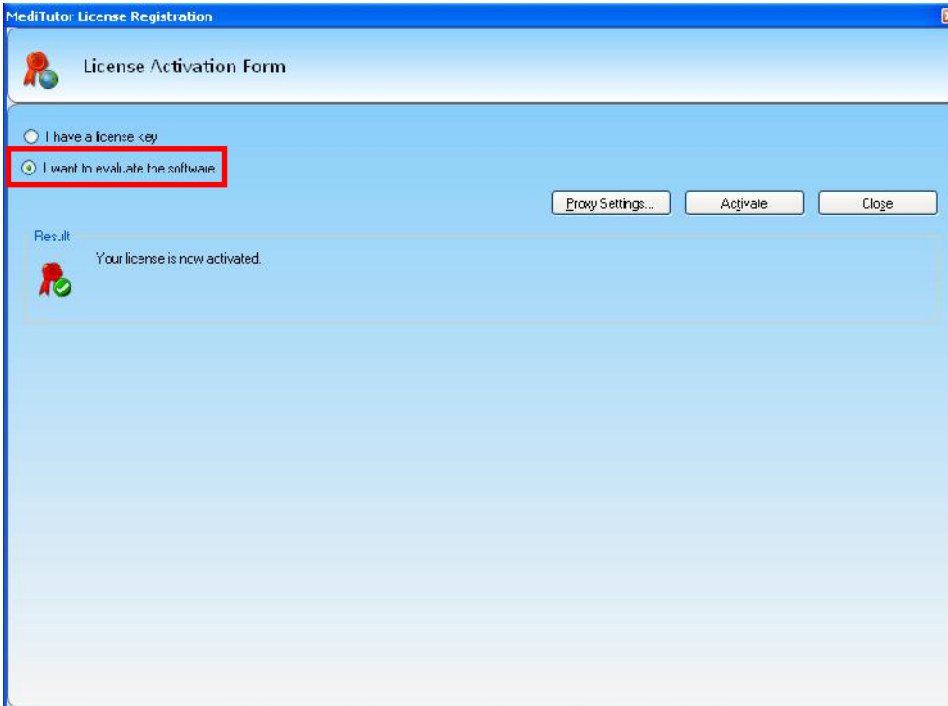


The screenshot shows the 'MediTutor: License Registration' window. The title bar is blue. Below the title bar is a header area with a ribbon icon and the text 'License Activation Form'. There are two radio buttons: 'I have a license key' (unselected) and 'I want to evaluate the software' (selected, highlighted with a red box). To the right of the radio buttons are three buttons: 'Proxy Settings...', 'Activate', and 'Close'. Below the radio buttons is a 'Result' section with a ribbon icon and the text 'Your license is now activated.'.

4. Insert the activation key you received from MediTouch

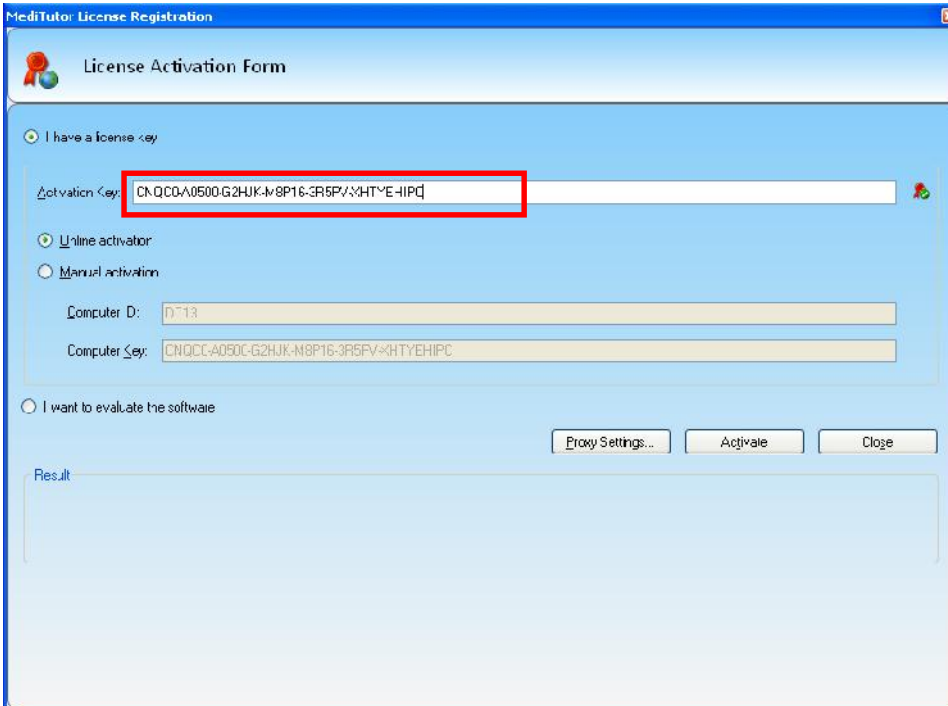


The screenshot shows the 'MediTutor: License Registration' window. The title bar is blue. Below the title bar is a header area with a ribbon icon and the text 'License Activation Form'. There are two radio buttons: 'I have a license key' (selected, highlighted with a red box) and 'I want to evaluate the software' (unselected). Below the radio buttons are three buttons: 'Proxy Settings...', 'Activate', and 'Close'. Below the buttons is a 'Result' section. The 'Activation Key' field is highlighted with a red box and contains the text 'CNQCC-A050C-G2HJK-M8P16-3R5FV-3HTYE-HIPC'. Below the 'Activation Key' field are two radio buttons: 'Online activator' (selected) and 'Manual activation' (unselected). Below the radio buttons are two text boxes: 'Computer ID' with the value 'D713' and 'Computer Key' with the value 'CNQCC-A050C-G2HJK-M8P16-3R5FV-3HTYE-HIPC'.

5. If you connect to web: Choose Online activation and press Activate

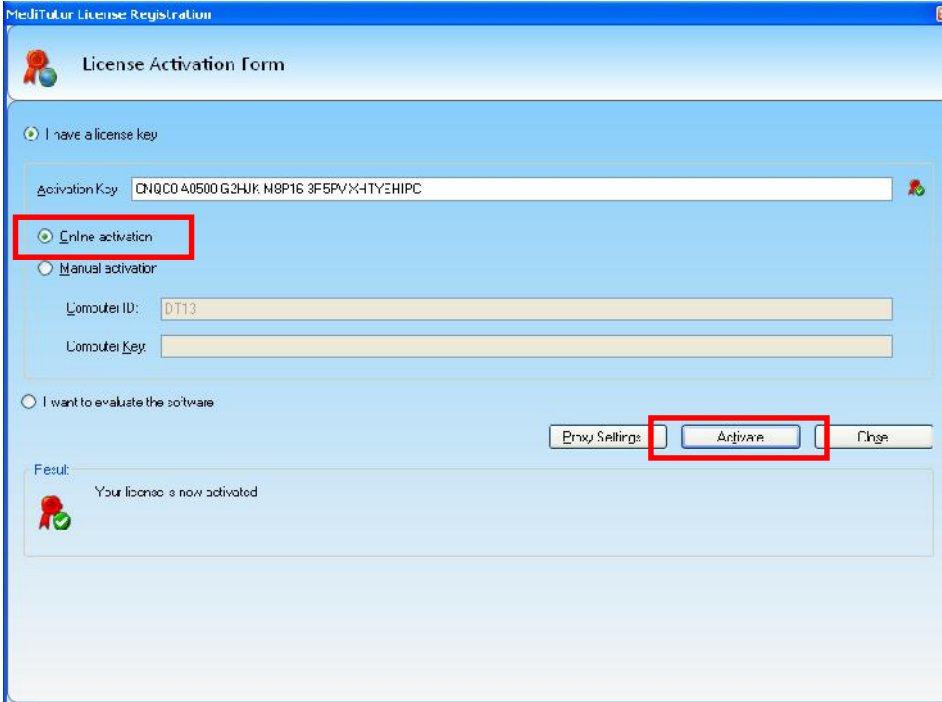

The screenshot shows the 'MediTutor License Registration' window with the 'License Activation Form'. The 'I have a license key' section is active. The 'Activation Key' field contains 'CNQCO 40500 G2HJK N8P16 3F5PVX4TYEHIPC'. The 'Online activation' radio button is selected and highlighted with a red box. Below it, the 'Manual activator' radio button is unselected. The 'Computer ID' field contains 'DT13' and the 'Computer Key' field is empty. At the bottom, the 'Activate' button is highlighted with a red box. The 'Result' section shows a green checkmark and the text 'Your license is now activated'.

6. If you disconnect from web: Choose Manual activation and press Activate

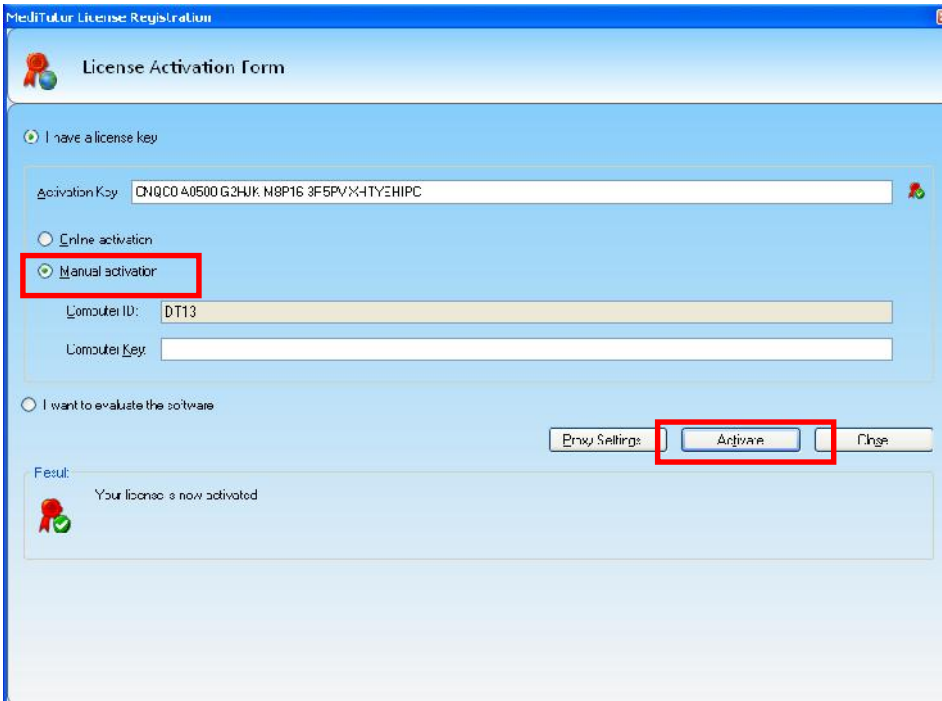

The screenshot shows the 'MediTutor License Registration' window with the 'License Activation Form'. The 'I have a license key' section is active. The 'Activation Key' field contains 'CNQCO 40500 G2HJK N8P16 3F5PVX4TYEHIPC'. The 'Manual activator' radio button is selected and highlighted with a red box. Below it, the 'Online activation' radio button is unselected. The 'Computer ID' field contains 'DT13' and the 'Computer Key' field is empty. At the bottom, the 'Activate' button is highlighted with a red box. The 'Result' section shows a green checkmark and the text 'Your license is now activated'.

7. Check your Expire date related to your license

