



BALANCETUTOR INSTALLATION GUIDE

- This booklet contains a comprehensive description of all the BalanceTutor installation stages.
- We have addressed all the necessary tools and instructions for a successful installation.
- Each and every stage must be followed in its specified order.
- All BalanceTutor assemblies must be performed by a MediTouch certified technician.
- Prior to the installation, the technician must varify that the facility is ready. Please comply with the **Installation Guidelines & Preperation** document (DO-15-01-16).
- For questions and comments email us at: support@meditouch.co.il

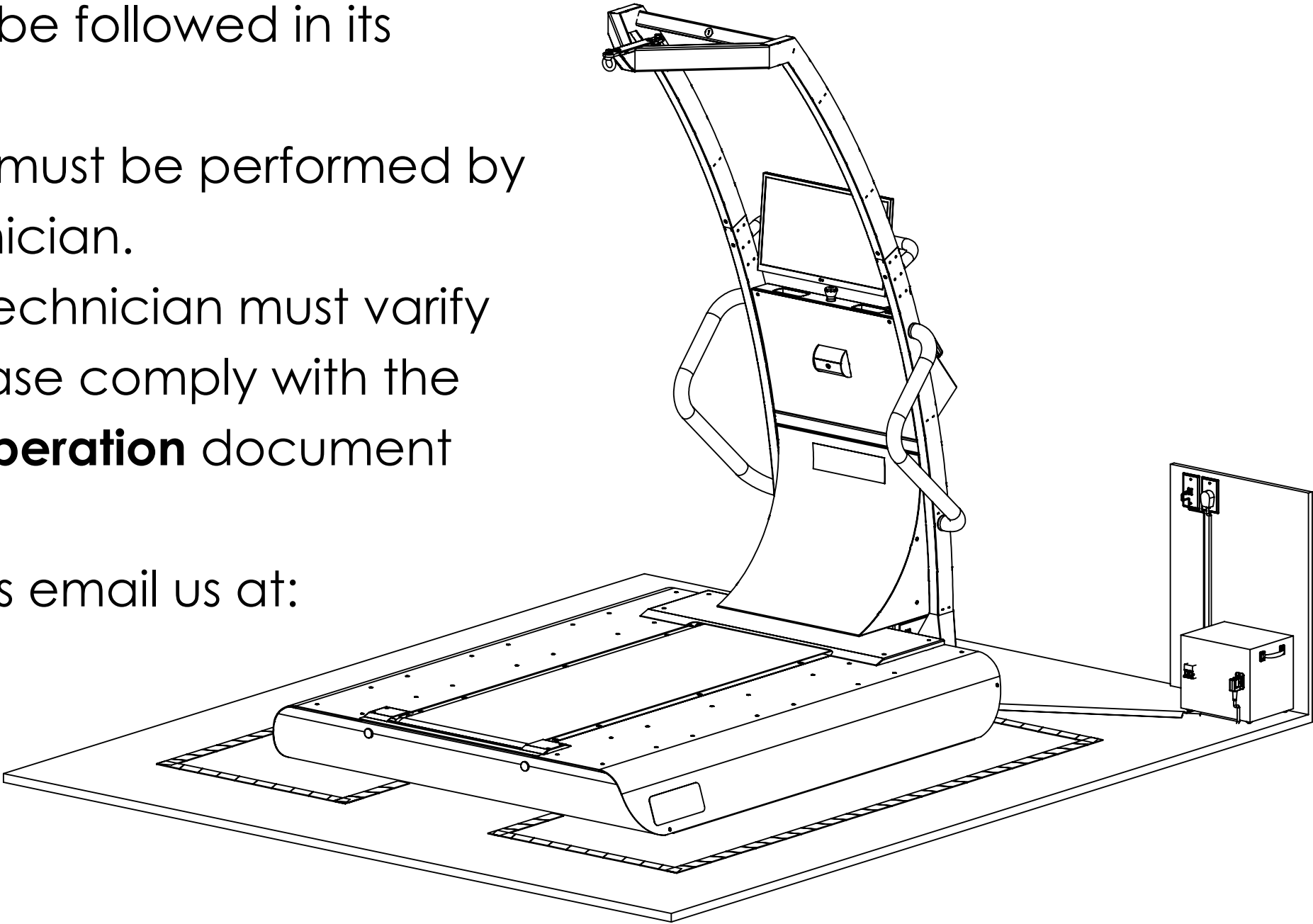


TOOL LIST


HAND PALLET TRUCK

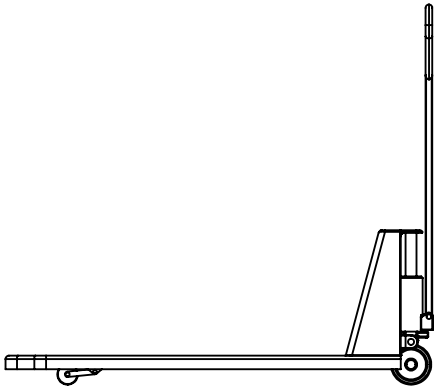

ETHERNET CABLE BLACK COLOR
3m & 5m

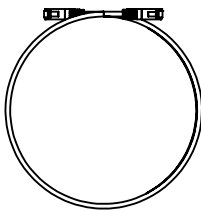


ELECTRICAL POWER PLUG
RIGHT ANGLE, 16A, BLACK
ACCORDING LOCAL STANDARDS

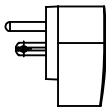

INSTALLATION REPORT
FIND INSIDE CRATE




SILICON
SPRAY

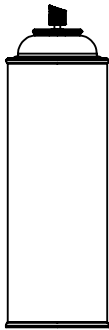

THREAD LOCKER
LOCTITE 243



2" YELLOW-BLACK
SAFETY DUCT TAPE

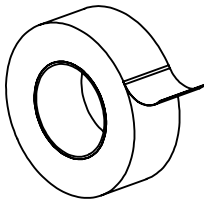

MEDITOUCH CERTIFIED
TECHNICIAN




GENERAL
WORKER




x2


REVERSIBLE RATCHET WRENCH
12, 17, 18, 19mm

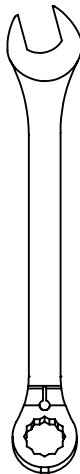


HEX ALLEN KEY
SIZE 2, 2.5, 4, 5, 6, 8mm

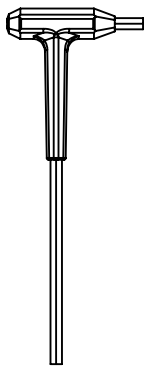


1/2" SOCKET
13, 17, 19mm

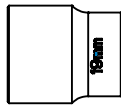


1/2" SOCKET
TUBE 17mm

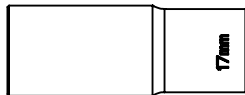


BIT TO SOCKET
1/2" ADAPTER

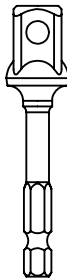

RATCHET
1/2"

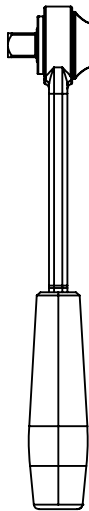


CABLE TIE 20cm
PACK OF 100

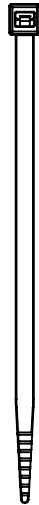


MINI STRAIGHT
SPIRIT LEVEL




CABLE
CUTTER

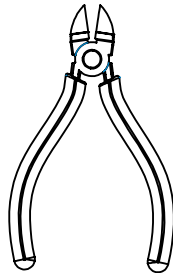

ELECTRIC
SCREWDRIVER

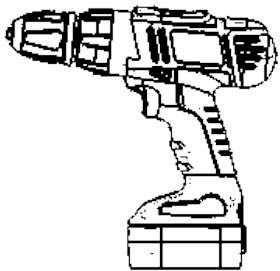


PHILLIPS
DRIVE BIT PH.2

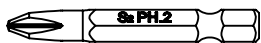


SNAP OFF
KNIFE




MEASURE TAPE

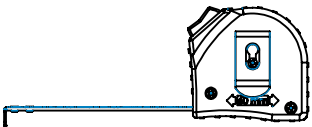

PHILLIPS SCREW DRIVER
PH.2

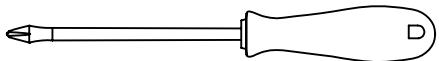

FLAT SCREW DRIVER
3mm and 10mm

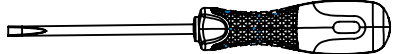

DOCUMENT NUMBER
DO-15-01-44_231122

DOCUMENT DESCRIPTION
Installation Guide

PROPRIETARY AND CONFIDENTIAL
THE INFORMATION CONTAINED IN THIS
DRAWING IS THE SOLE PROPERTY OF
MEDITOUCH LTD. ANY REPRODUCTION IN
PART OR AS A WHOLE WITHOUT THE
WRITTEN PERMISSION OF MEDITOUCH LTD.
IS PROHIBITED.

Assembly Drawing

THIRD
ANGLE
PROJECTION



ALL DIMENSIONS IN
mm



MediTouch

1 NIRIM STREET, TNUVOT, ISRAEL.

DRAWN BY
Ziv


SCALE
1:15

SHEET SIZE
A4

PAGE
2 of 26

CRATE DISASSEMBLY

ELECTRIC
SCREWDRIVER

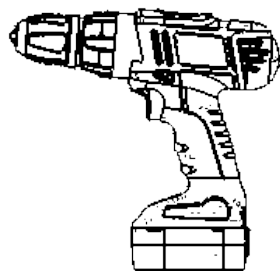


PHILLIPS
DRIVE BIT PH.2



SNAP OFF
KNIFE




HAND PALLET
TRUCK

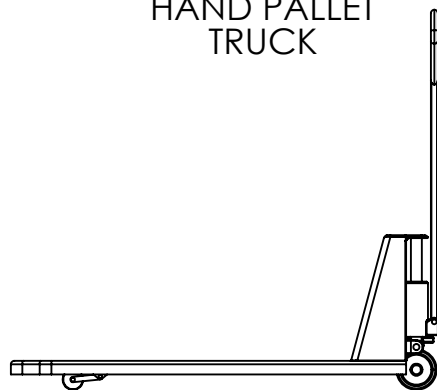

CONFIRM ALL ITEMS LISTED IN THE PACKING LIST
AND IDENTIFIED FROM THE CRATE



DO-15-01-10

~650Kg
~ 1430 Lbs

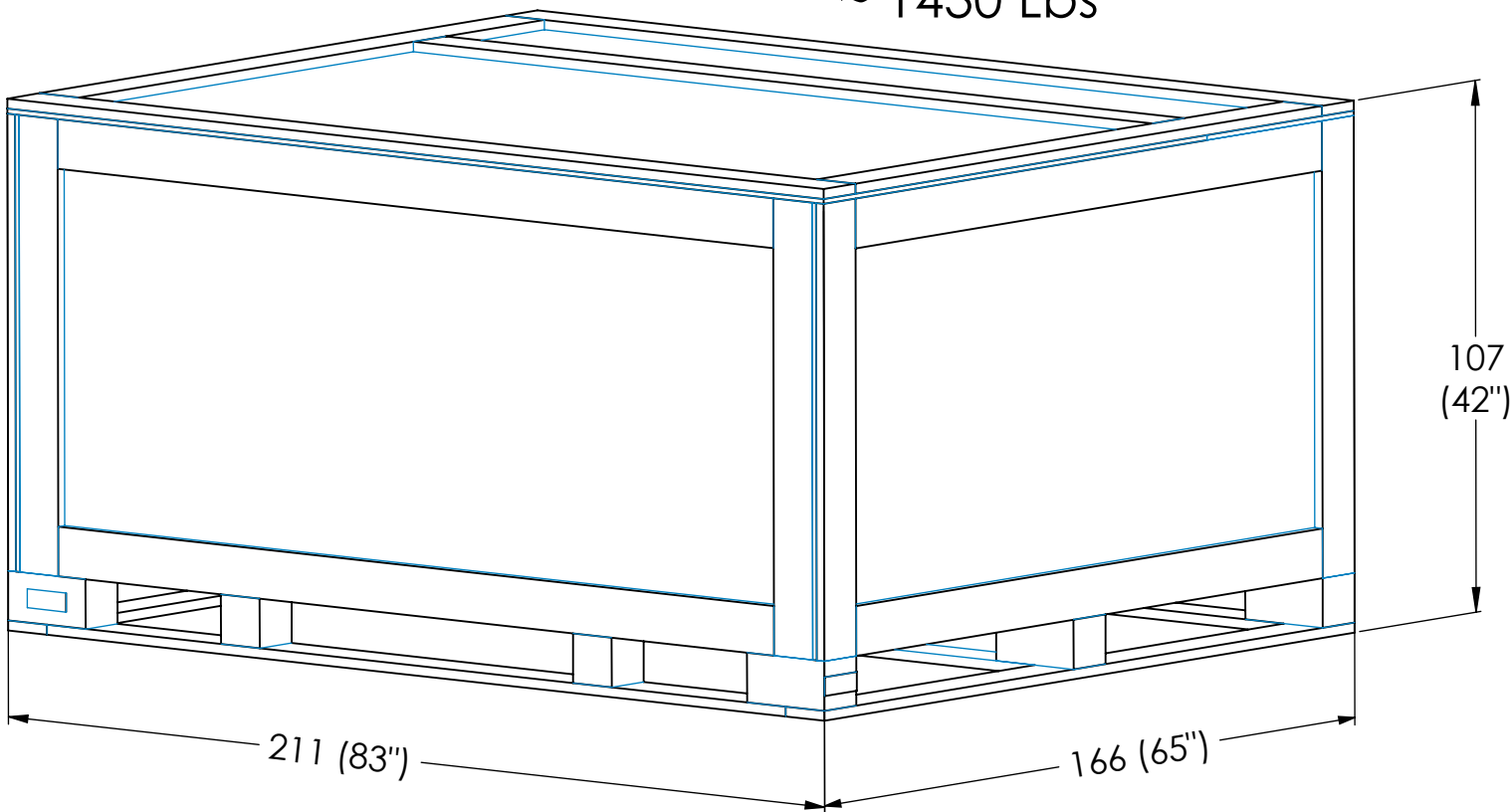





DOCUMENT NUMBER DO-15-01-44_231122		PROPRIETARY AND CONFIDENTIAL THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.		Assembly Drawing		 MediTouch 1 NIRIM STREET, TNUVOT, ISRAEL.	DRAWN BY Ziv	SHEET SIZE A4
DOCUMENT DESCRIPTION Installation Guide				THIRD ANGLE PROJECTION	ALL DIMENSIONS IN mm		SCALE 1:15	PAGE 3 of 26

BASE PLACEMENT

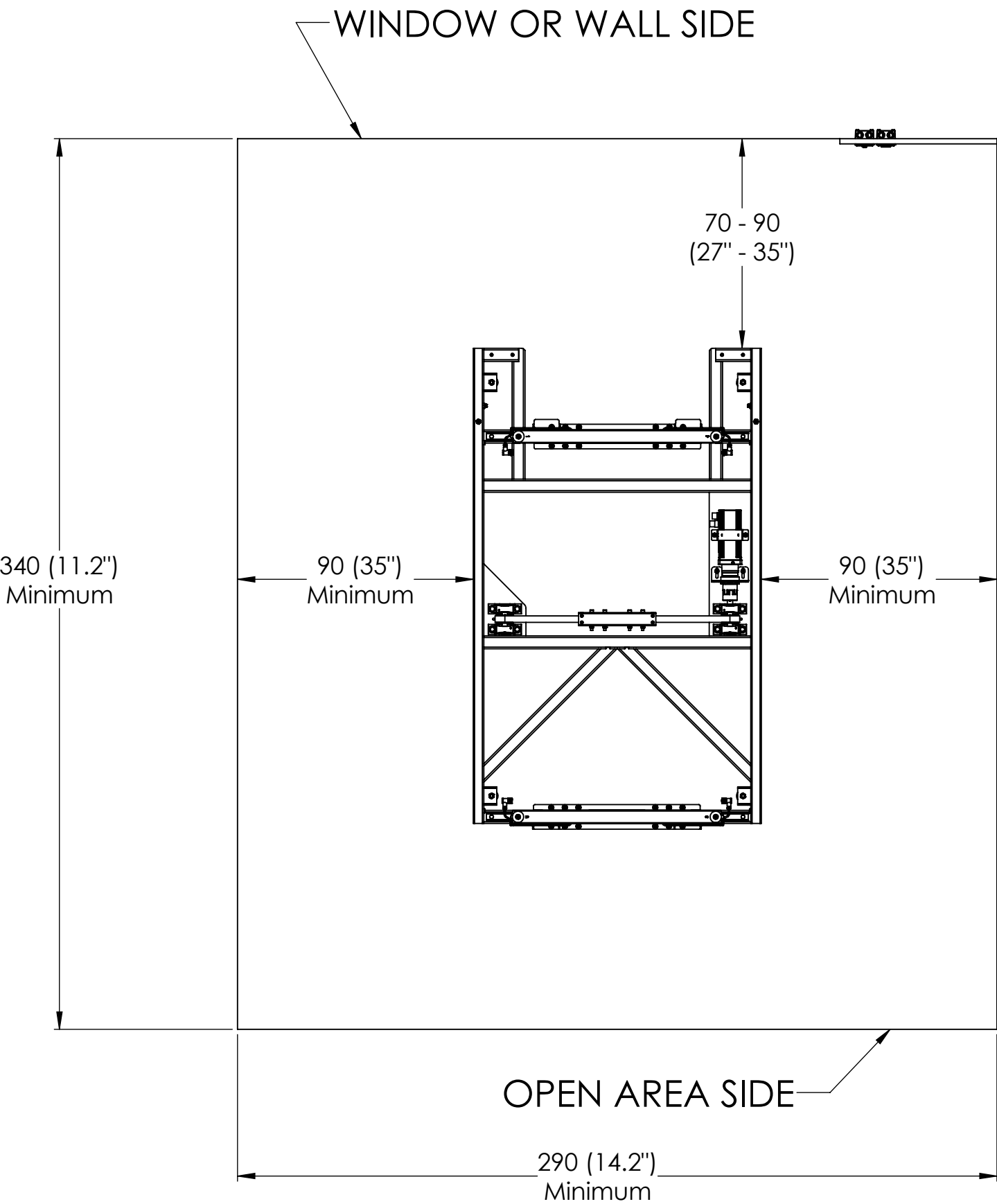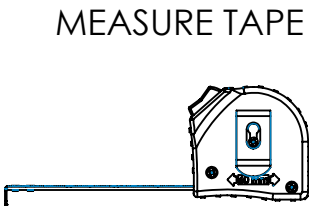




DOCUMENT NUMBER DO-15-01-44_231122	PROPRIETARY AND CONFIDENTIAL THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.	Assembly Drawing		 MediTouch 1 NIRIM STREET, TNUVOT, ISRAEL.	DRAWN BY Ziv	SHEET SIZE A4
DOCUMENT DESCRIPTION Installation Guide		THIRD ANGLE PROJECTION 	ALL DIMENSIONS IN mm		SCALE 1:20	PAGE 4 of 26

MOTOR & SENSOR CABLES ASSEMBLY

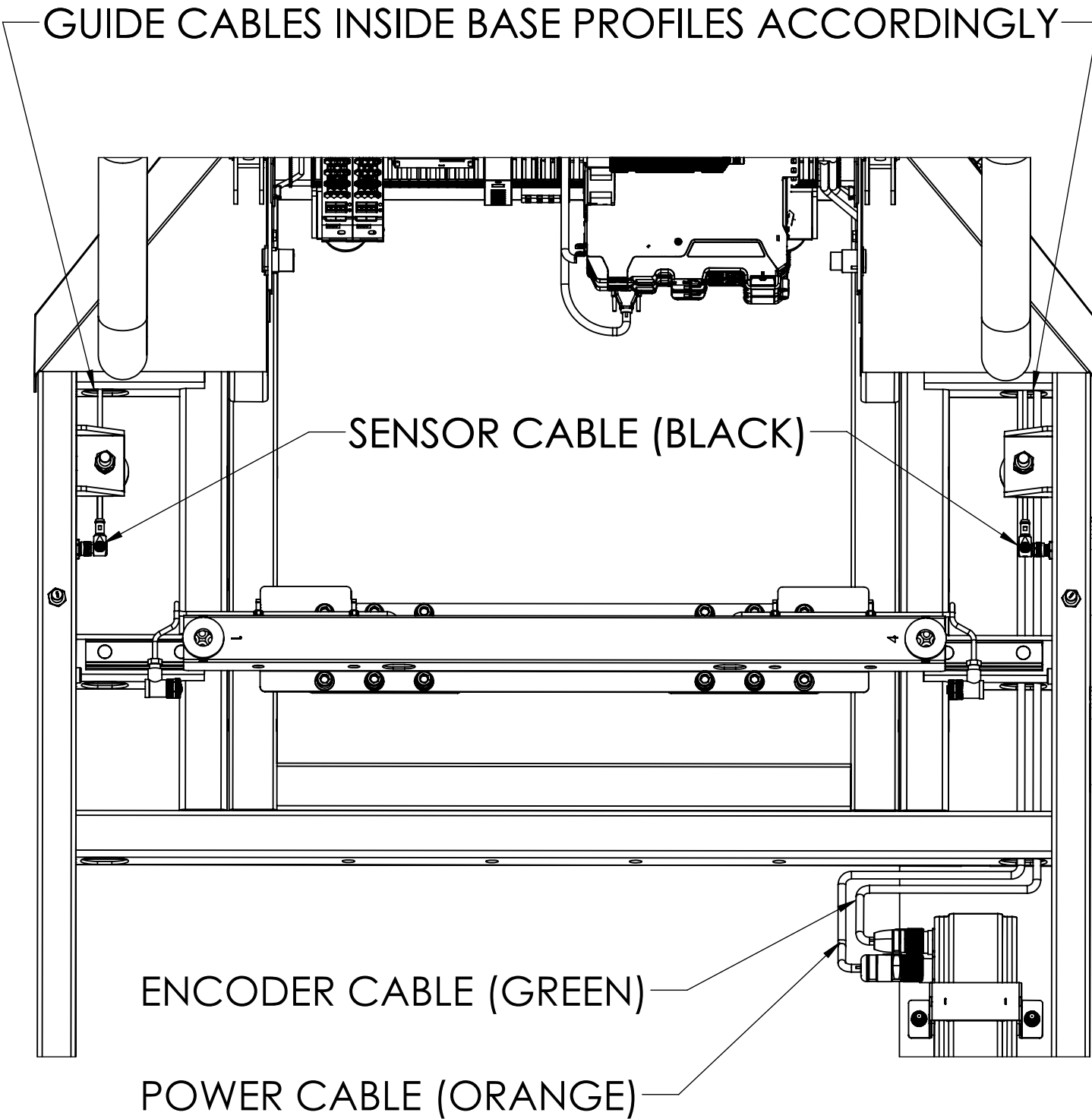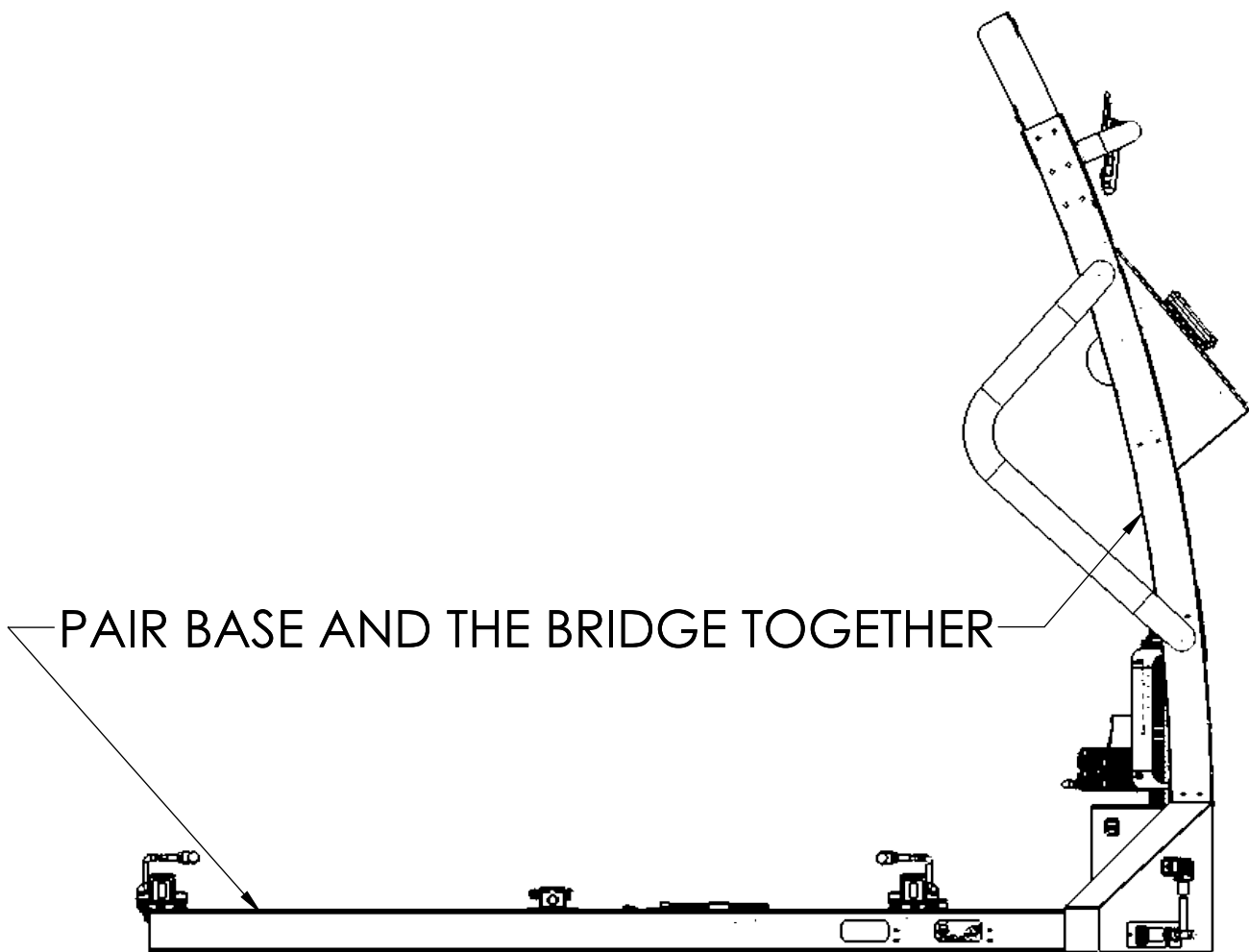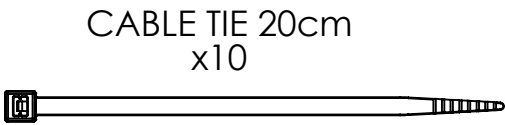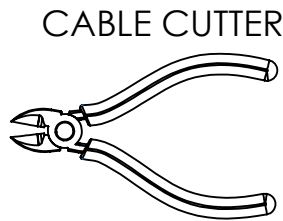

DOCUMENT NUMBER DO-15-01-44_231122	PROPRIETARY AND CONFIDENTIAL THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.	Assembly Drawing		 MediTouch 1 NIRIM STREET, TNUVOT, ISRAEL.	DRAWN BY Ziv	SHEET SIZE A4
DOCUMENT DESCRIPTION Installation Guide		THIRD ANGLE PROJECTION 	ALL DIMENSIONS IN mm		SCALE 1:14	PAGE 5 of 26

BASE & BRIDGE ASSEMBLY

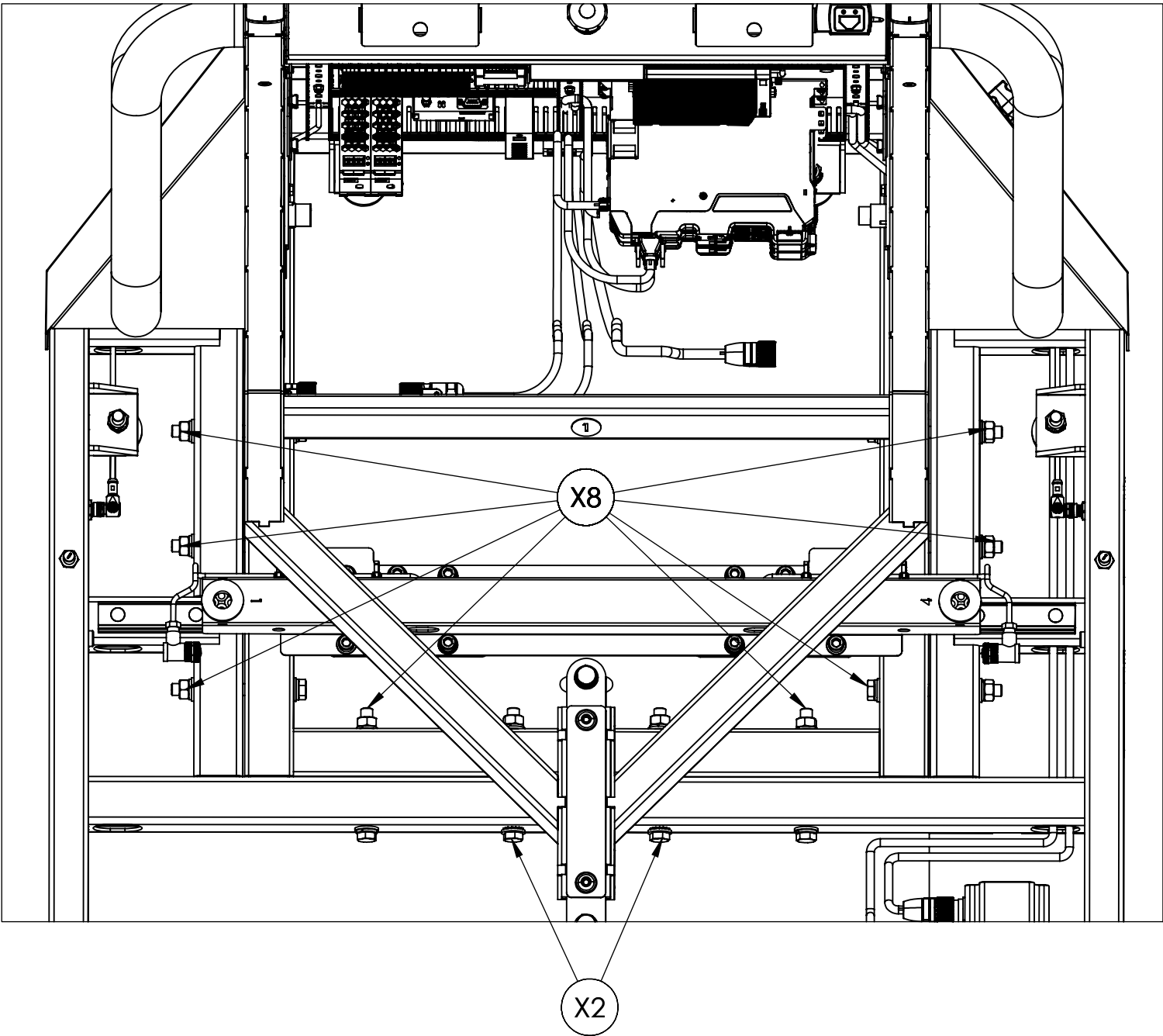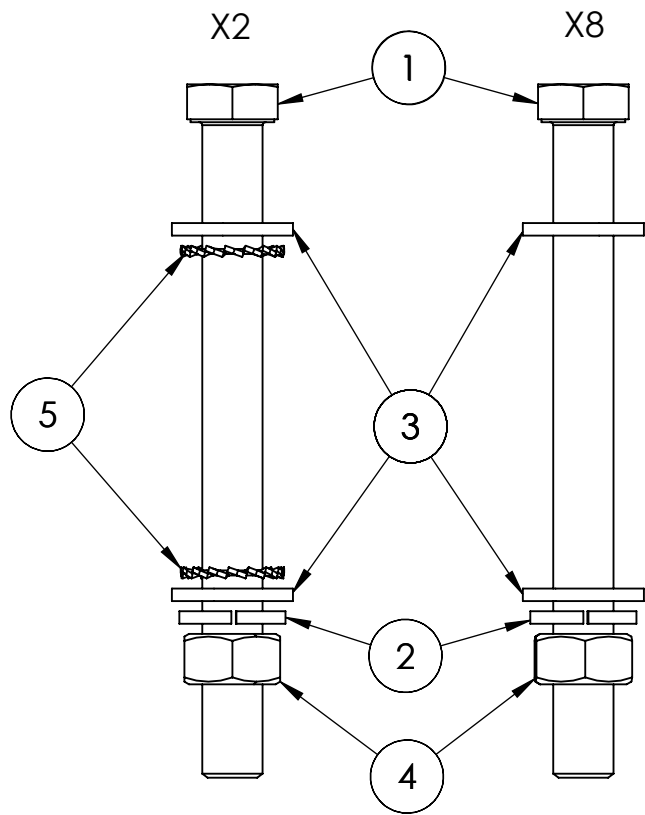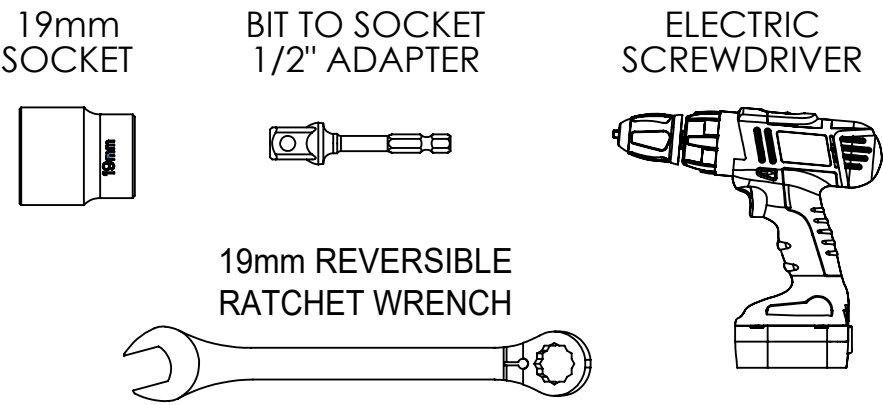


ITEM NO.	PART NUMBER	DESCRIPTION	QTY.
1	DS-02-01-41-D	Hexagon Head Bolt M12X130 DIN931 Steel ZP 8.8	10
2	DS-02-01-30-D	Washer Spring Lock M12 DIN127-B Spring steel	10
3	DS-02-01-31-E	Washer M12 DIN125-1A Steel 140HV ZP	40
4	DS-02-01-38-B	Nut M12 DIN934 Steel ZP	10
5	DS-02-01-42-A	Washer Serrated Lock M12 DIN6798 ZP	4




DOCUMENT NUMBER DO-15-01-44_231122	PROPRIETARY AND CONFIDENTIAL THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.	Assembly Drawing THIRD ANGLE PROJECTION ALL DIMENSIONS IN mm	 1 NIRIM STREET, TNUVOT, ISRAEL.	DRAWN BY Ziv	SHEET SIZE A4
DOCUMENT DESCRIPTION Installation Guide				SCALE 1:14	PAGE 6 of 26

LEVELING


17mm
SOCKET

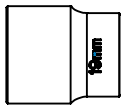


RATCHET
1/2"

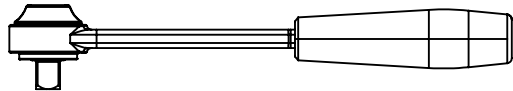

MINI STRAIGHT
SPIRIT LEVEL

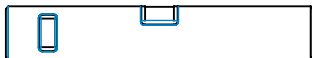


12mm REVERSIBLE
RATCHET WRENCH

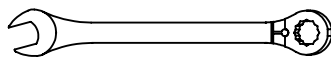

- 1. ELEVATE THE DEVICE 2-3MM FROM GROUND
- 2. LEVEL THE DEVICE IN X AND Y AXIS USING SPIRIT LEVEL

LEVELING FOOT X6

STRAIGHT SPIRIT LEVEL

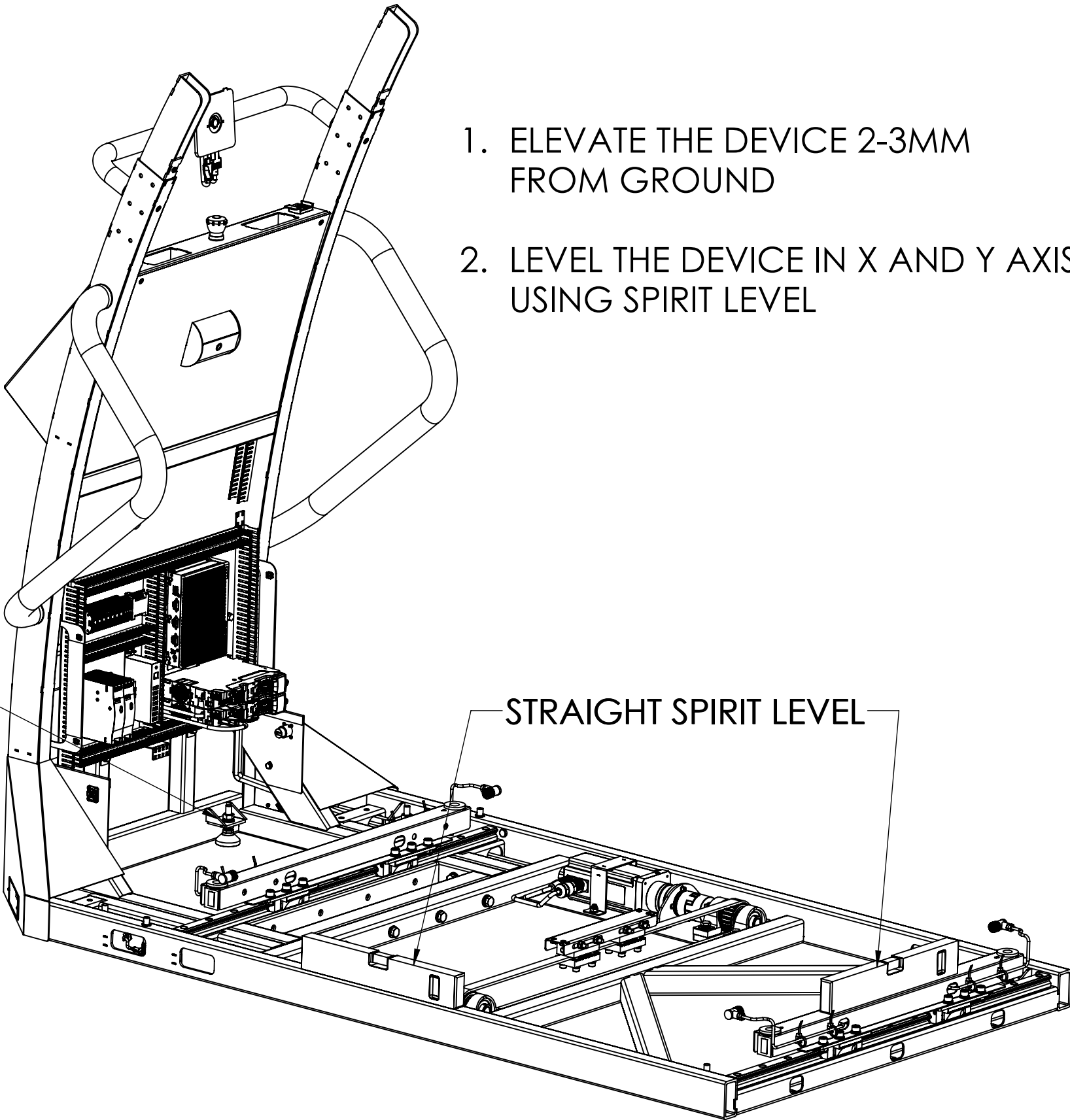




DOCUMENT NUMBER DO-15-01-44_231122	PROPRIETARY AND CONFIDENTIAL THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.	Assembly Drawing	 1 NIRIM STREET, TNUVOT, ISRAEL.	DRAWN BY Ziv	SHEET SIZE A4
DOCUMENT DESCRIPTION Installation Guide	THIRD ANGLE PROJECTION 	ALL DIMENSIONS IN mm		SCALE 1:10	PAGE 7 of 26

BASE & PLATFORM ASSEMBLY

13mm
SOCKET



RATCHET
1/2"

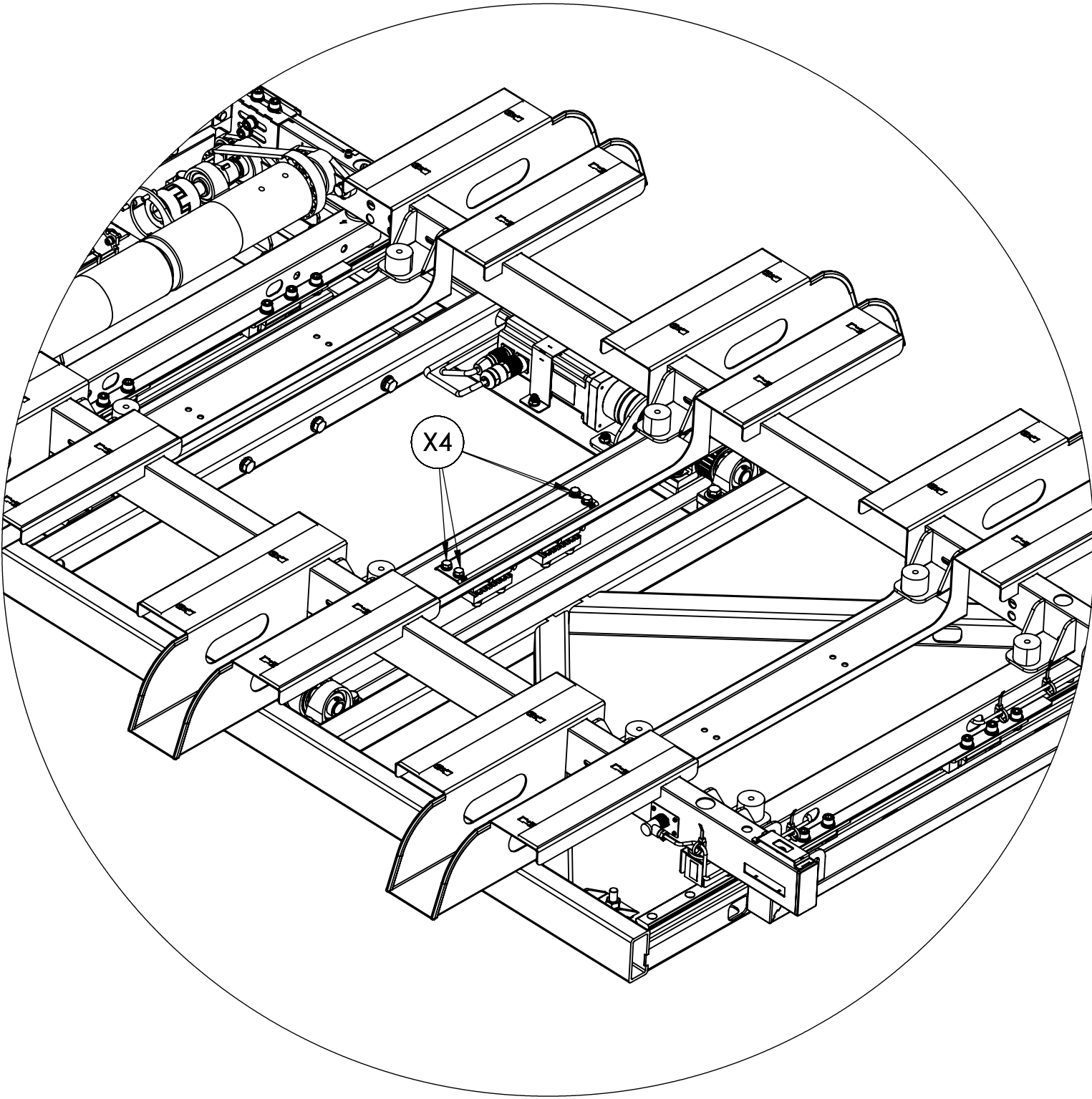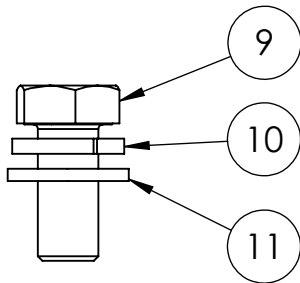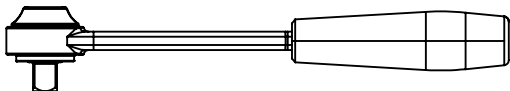

ITEM NO.	PART NUMBER	DESCRIPTION	QTY.
9	DS-02-01-32-B	Hexagon Head Bolt M8X18 DIN933 Steel ZP 8.8	4
10	DS-02-01-30-A	Washer Spring Lock M8 DIN127-B Spring steel	4
11	DS-02-01-31-A	Washer M8 DIN125-1A Steel 140HV ZP	4



DOCUMENT NUMBER
DO-15-01-44_231122

DOCUMENT DESCRIPTION
Installation Guide

PROPRIETARY AND CONFIDENTIAL

THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.

Assembly Drawing

THIRD
ANGLE
PROJECTION



ALL DIMENSIONS IN
mm



MediTouch

1 NIRIM STREET, TNUVOT, ISRAEL.

DRAWN BY
Ziv


SCALE
1:10

SHEET SIZE
A4



PAGE
8 of 26

BACK PULLEY ASSEMBLY


17mm
SOCKET

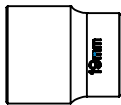

RATCHET
1/2"

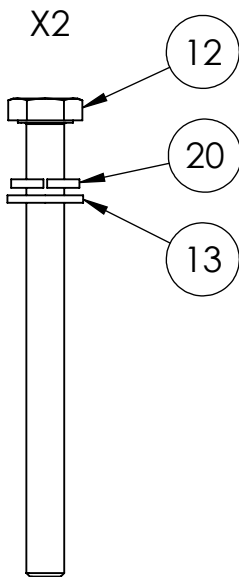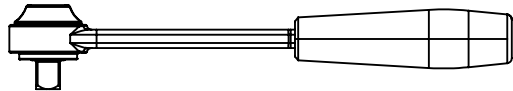

- 1. PLACE BACK PULLEY
- 2. DO NOT TIGHT THE BOLTS, KEEP BELT LOOSE

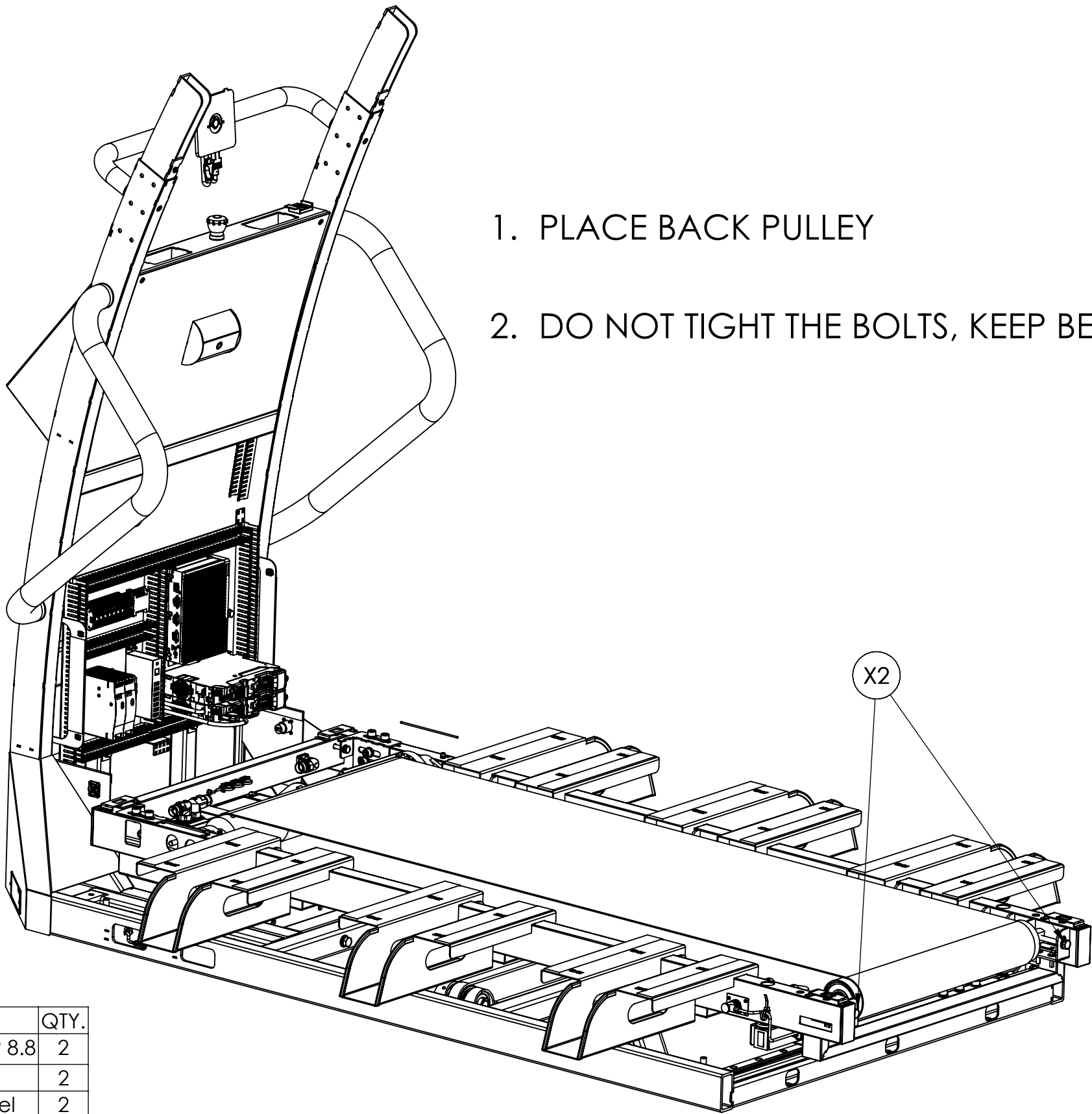


ITEM NO.	PART NUMBER	DESCRIPTION	QTY.
12	DS-02-01-36-C	Hexagon Head Bolt M10X120 DIN933 Steel ZP 8.8	2
13	DS-02-01-31-B	Washer M10 DIN125-1A Steel 140HV ZP	2
20	DS-02-01-30-B	Washer Spring Lock M10 DIN-127-B Spring steel	2



DOCUMENT NUMBER DO-15-01-44_231122	PROPRIETARY AND CONFIDENTIAL THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.	Assembly Drawing THIRD ANGLE PROJECTION	ALL DIMENSIONS IN mm	 MediTouch 1 NIRIM STREET, TNUVOT, ISRAEL.	DRAWN BY Ziv SCALE 1:10	SHEET SIZE A4 PAGE 9 of 26
---------------------------------------	--	--	--------------------------------	--	---	--

LOAD CELLS CONNECTORS


- 1. ASSEMBLE 4 CONNECTORS ACCORDING SPECIFIED OERIENTATION
- 2. SECURE SENSOR CABLES WITH SUPPLIED CABLE TIES

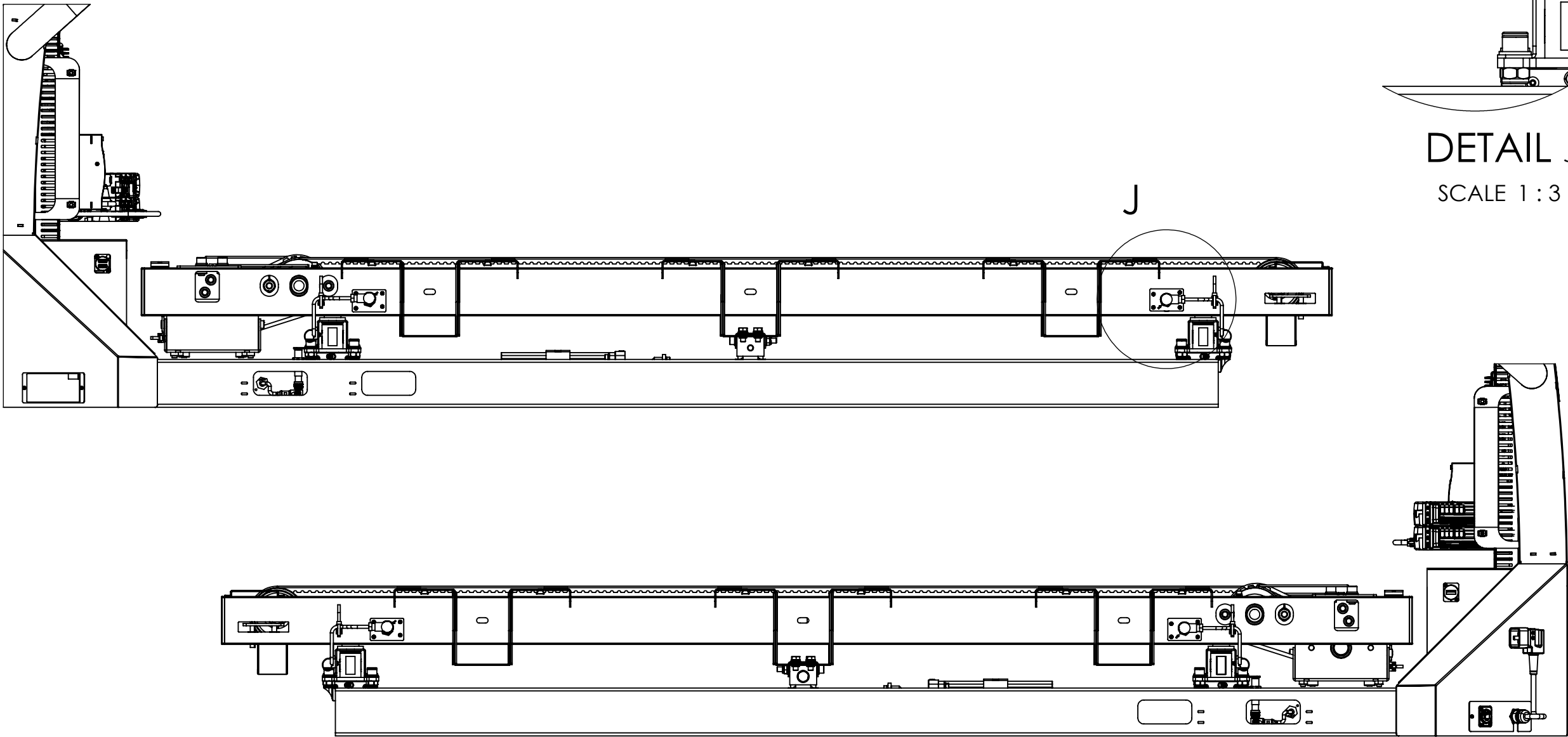



DOCUMENT NUMBER DO-15-01-44_231122	DOCUMENT DESCRIPTION Installation Guide	PROPRIETARY AND CONFIDENTIAL THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.	Assembly Drawing	THIRD ANGLE PROJECTION	ALL DIMENSIONS IN mm	 1 NIRIM STREET, TNUVOT, ISRAEL.	DRAWN BY Ziv	SHEET SIZE A4
							SCALE 1:8	PAGE 10 of 26

RUNNING BOARD ASSEMBLY


SILICON SPRAY

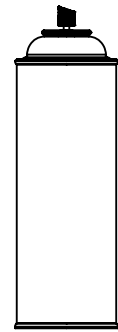

HEX ALLEN KEY
SIZE 6mm

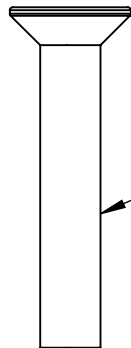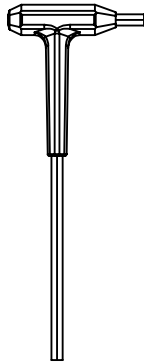

8

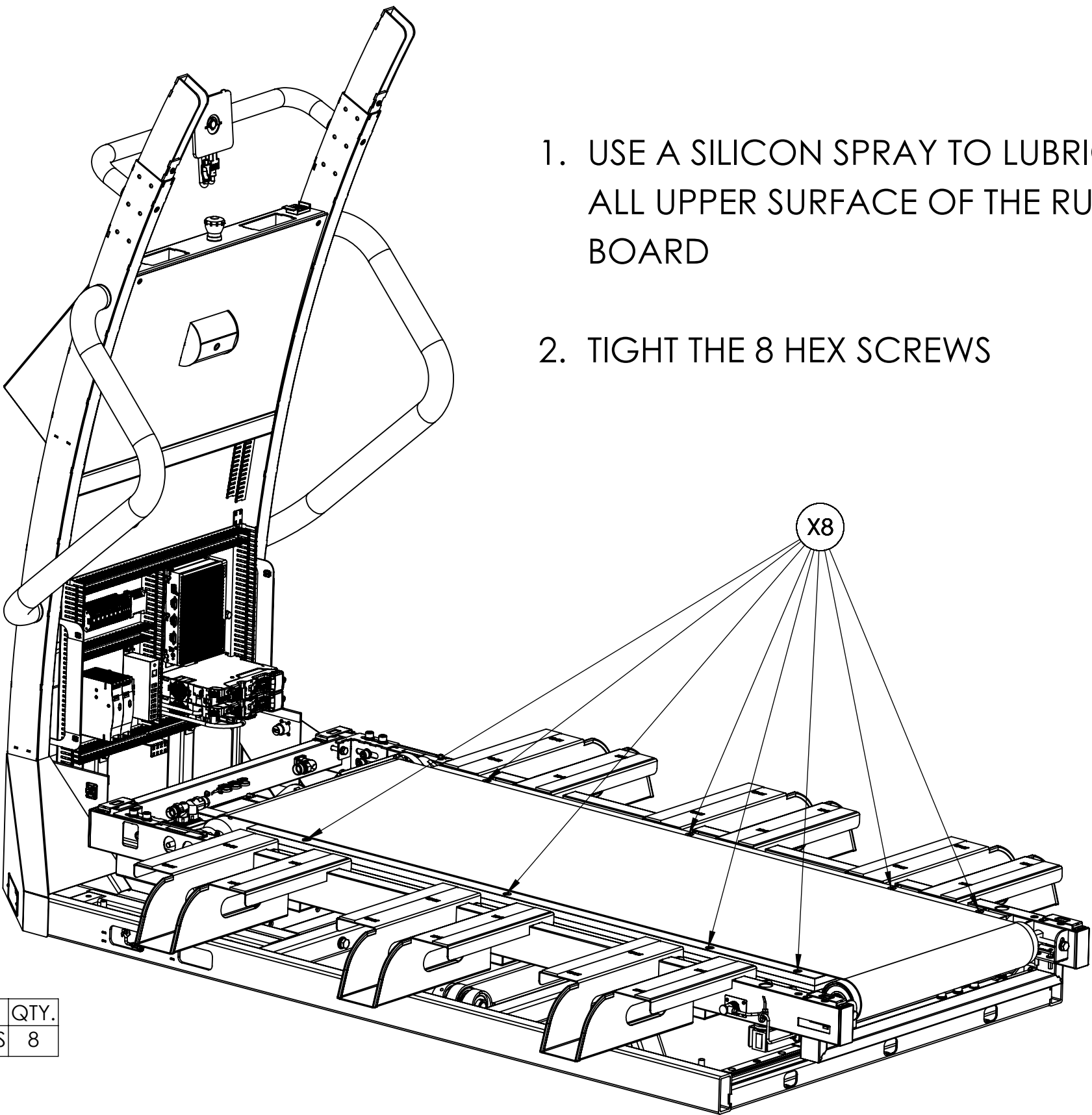

1. USE A SILICON SPRAY TO LUBRICATE ALL UPPER SURFACE OF THE RUNNING BOARD
2. TIGHT THE 8 HEX SCREWS

ITEM NO.	PART NUMBER	DESCRIPTION	QTY.
8	DS-02-01-43-A	Countersunk Hexagon Socket M8x45 DIN7991 SS	8



DOCUMENT NUMBER
DO-15-01-44_231122

DOCUMENT DESCRIPTION
Installation Guide

PROPRIETARY AND CONFIDENTIAL

THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.

Assembly Drawing

THIRD
ANGLE
PROJECTION



ALL DIMENSIONS IN
mm



MediTouch

1 NIRIM STREET, TNUVOT, ISRAEL.

DRAWN BY
Ziv


SCALE
1:10

SHEET SIZE
A4



PAGE
11 of 26

TIGHTNING BACK PULLEY

17mm
SOCKET

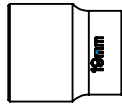

RATCHET
1/2"

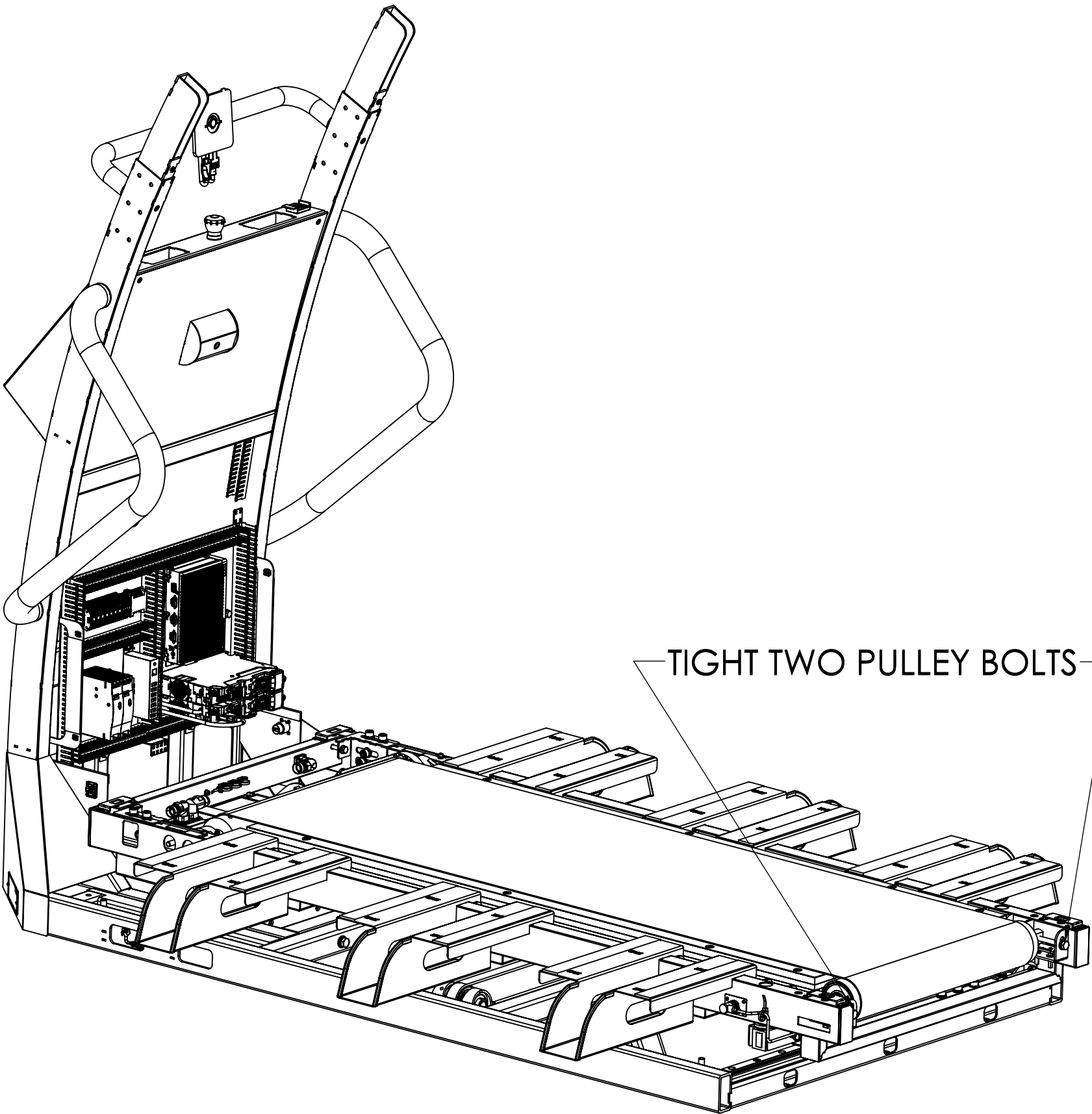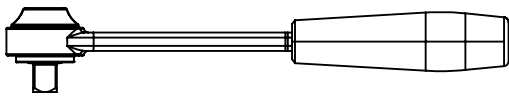

TIGHT TWO PULLEY BOLTS



DOCUMENT NUMBER DO-15-01-44_231122	PROPRIETARY AND CONFIDENTIAL THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.	Assembly Drawing	 1 NIRIM STREET, TNUVOT, ISRAEL.	DRAWN BY Ziv	SHEET SIZE A4
DOCUMENT DESCRIPTION Installation Guide				SCALE 1:10	PAGE 12 of 26



MOTOR HOUSE CABLES ASSEMBLY

X3 CABLE TIES ON FRAME SIDE
X3 CABLE TIES ON MOTOR HOUSE SIDE

AFTER FINISHING CABLE ASSEMBLY,
SECURE ALL CABLES WITH CABLE TIES.

CABLE TIE 20cm
PACK OF 100

CABLE
CUTTER

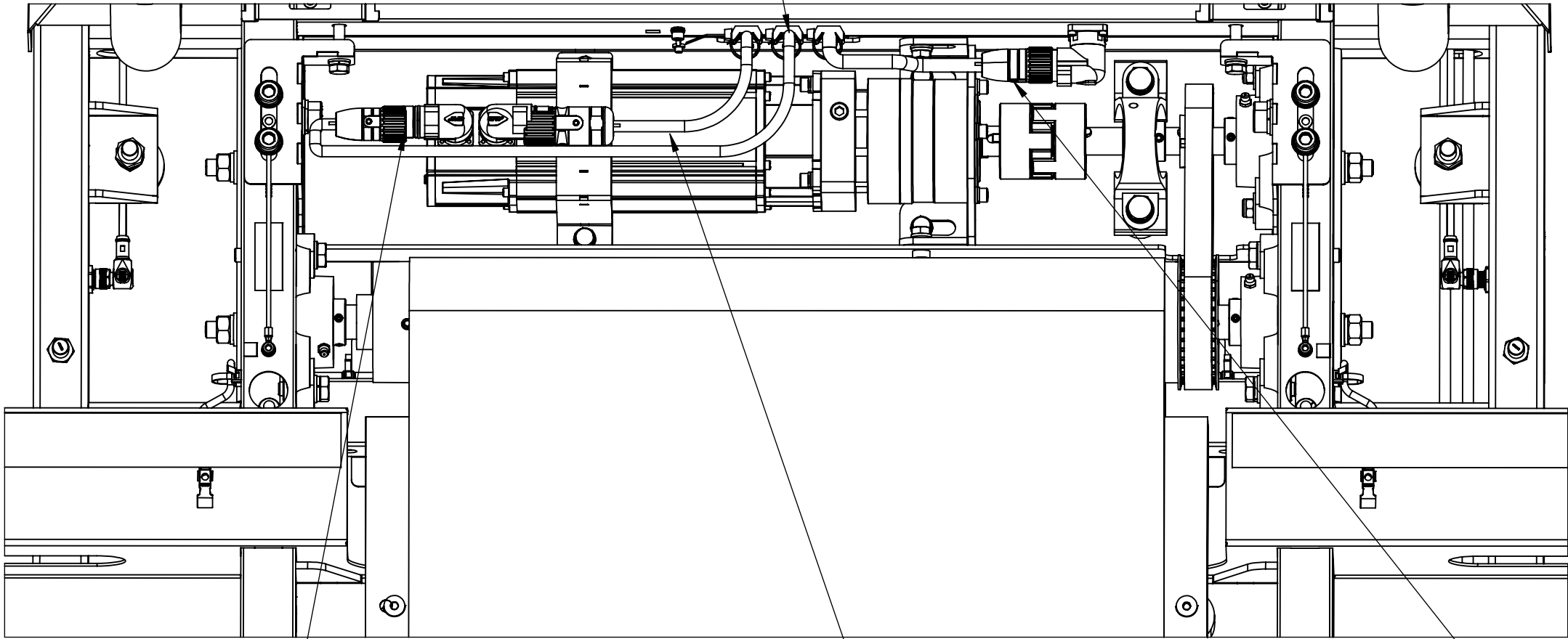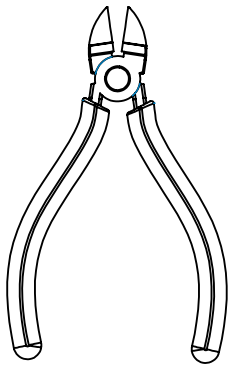

ENCODER CONNECTOR
GREEN CABLE

POWER CONNECTOR
ORANGE CABLE

LOAD CELLS CONNECTOR
BLACK CABLE



DOCUMENT NUMBER
DO-15-01-44_231122
DOCUMENT DESCRIPTION
Installation Guide

PROPRIETARY AND CONFIDENTIAL
THE INFORMATION CONTAINED IN THIS
DRAWING IS THE SOLE PROPERTY OF
MEDITOUCH LTD. ANY REPRODUCTION IN
PART OR AS A WHOLE WITHOUT THE
WRITTEN PERMISSION OF MEDITOUCH LTD.
IS PROHIBITED.

Assembly Drawing

THIRD
ANGLE
PROJECTION



ALL DIMENSIONS IN
mm



MediTouch

1 NIRIM STREET, TNUVOT, ISRAEL.

DRAWN BY
Ziv



SCALE
1:4

SHEET SIZE
A4



PAGE
13 of 26

UPPER BRIDGE ASSEMBLY

HEX ALLEN KEY
SIZE 5mm

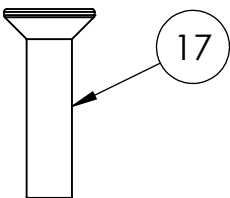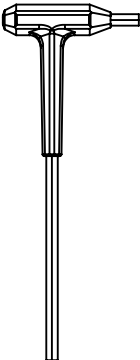

ITEM NO.	PART NUMBER	DESCRIPTION	QTY.
17	DS-02-01-40-A	Countersunk Hexagon Socket M6x25 DIN7991 SS	4

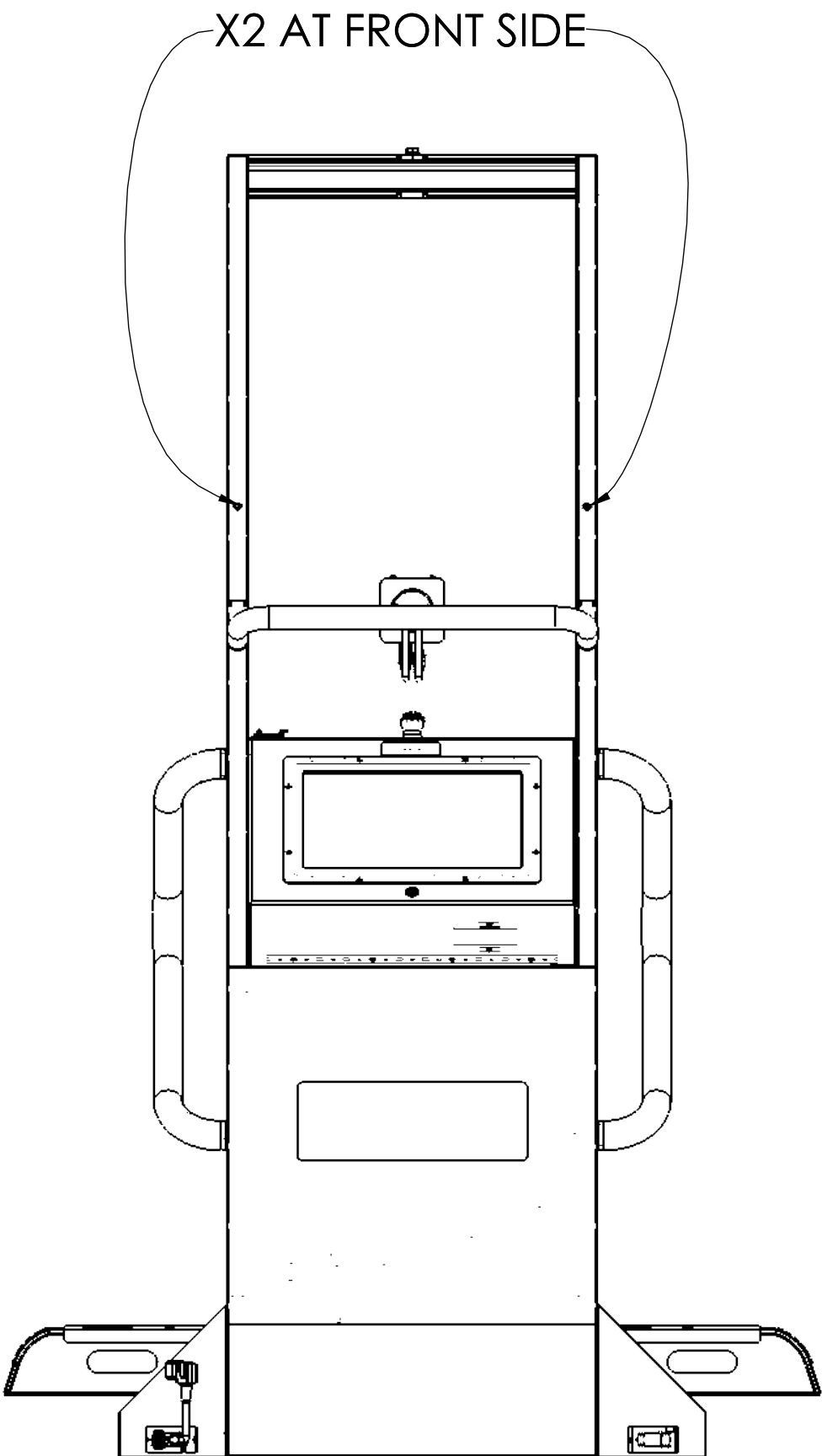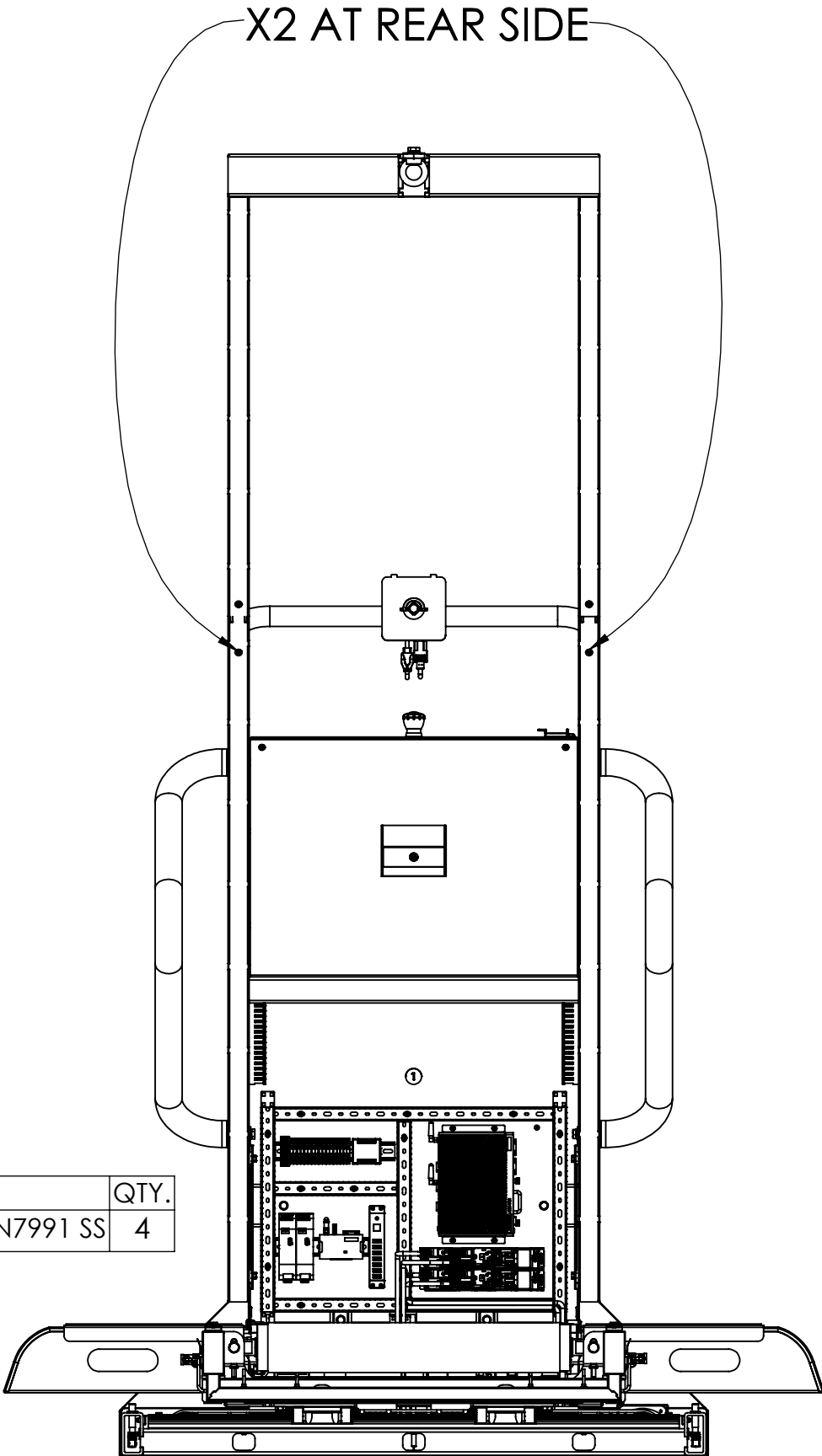

DOCUMENT NUMBER
DO-15-01-44_231122

DOCUMENT DESCRIPTION
Installation Guide

PROPRIETARY AND CONFIDENTIAL

THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.

Assembly Drawing

THIRD
ANGLE
PROJECTION



ALL DIMENSIONS IN
mm



MediTouch

1 NIRIM STREET, TNUVOT, ISRAEL.

DRAWN BY
Ziv



SCALE
1:12

SHEET SIZE
A4

PAGE
14 of 26

MONITOR ASSEMBLY

- 1. CLICK THE SCREEN ON SCREEN HOLDER
- 2. CONNECT HDMI CABLE
- 3. CONNECT POWER CABLE

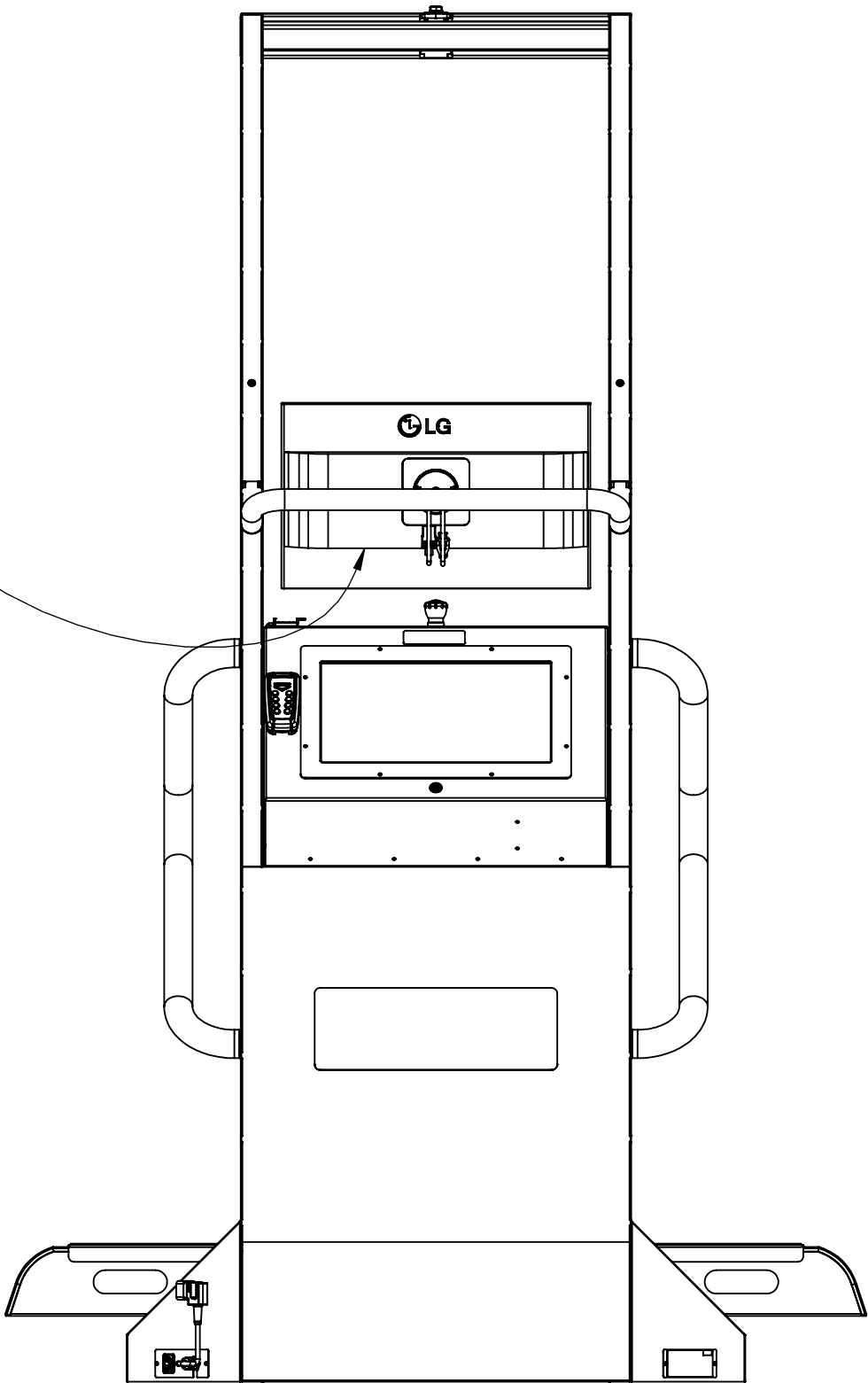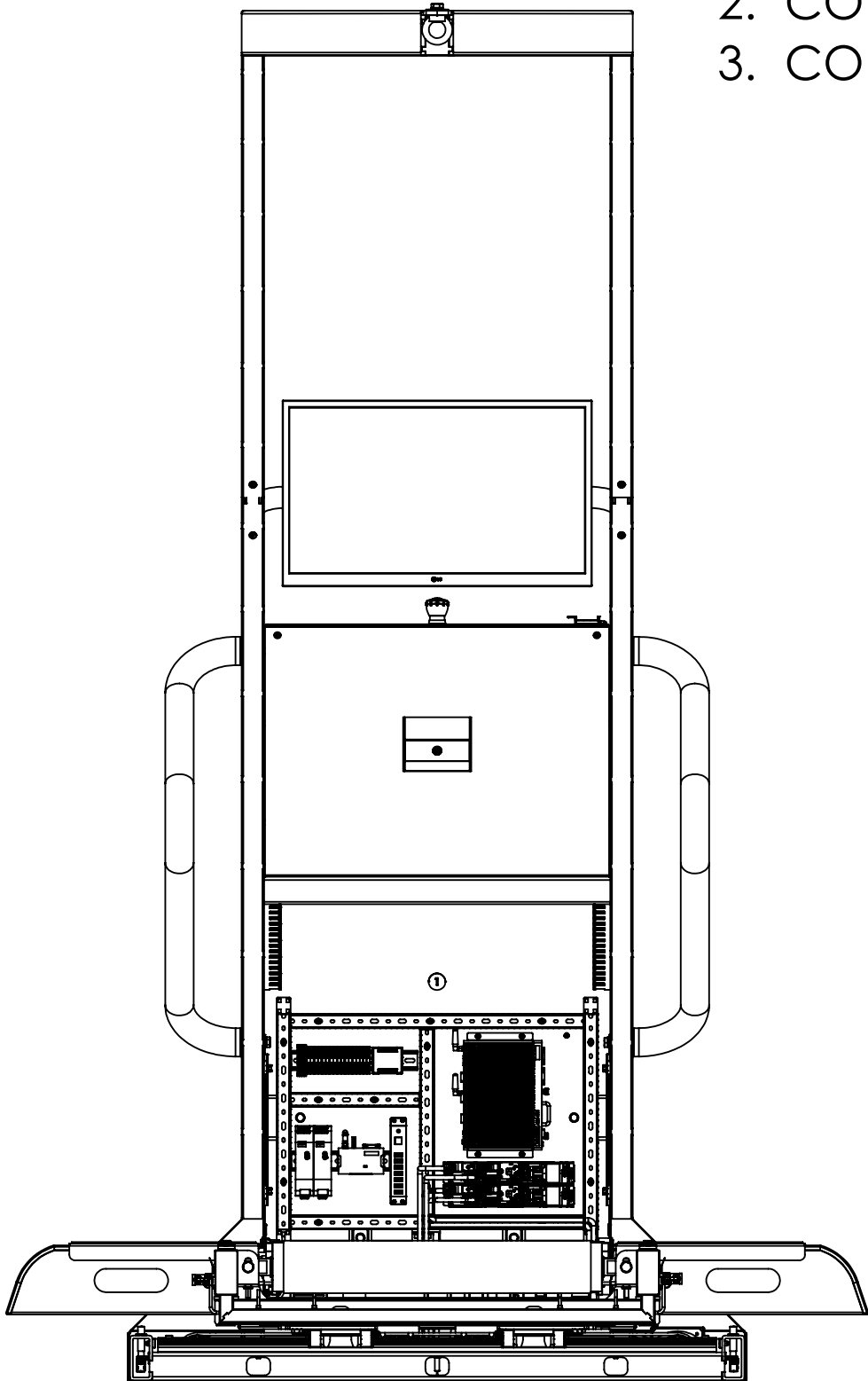

DOCUMENT NUMBER
DO-15-01-44_231122

DOCUMENT DESCRIPTION
Installation Guide

PROPRIETARY AND CONFIDENTIAL
THE INFORMATION CONTAINED IN THIS
DRAWING IS THE SOLE PROPERTY OF
MEDITOUCH LTD. ANY REPRODUCTION IN
PART OR AS A WHOLE WITHOUT THE
WRITTEN PERMISSION OF MEDITOUCH LTD.
IS PROHIBITED.

Assembly Drawing

THIRD
ANGLE
PROJECTION



ALL DIMENSIONS IN
mm



1 NIRIM STREET, TNUVOT, ISRAEL.

DRAWN BY
Ziv


SCALE
1:12

SHEET SIZE
A4


PAGE
15 of 26

TRANSFORMER & ETHERNET


PHILLIPS SCREW DRIVER
PH.2

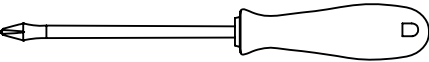


FLAT SCREW DRIVER
3mm

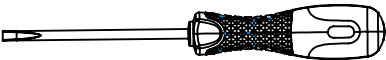


CABLE TIE 20cm

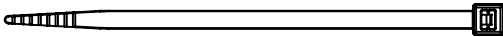


CABLE CUTTER

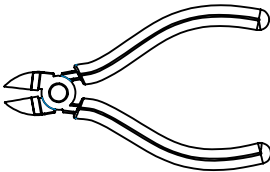



1. CUT FLOOR CABLE PROTECTOR
ACCORDING CORRECT LENGTH
2. CLEAN FLOOR BEFORE ATTACHING
THE FLOOR CABLE PROTECTOR

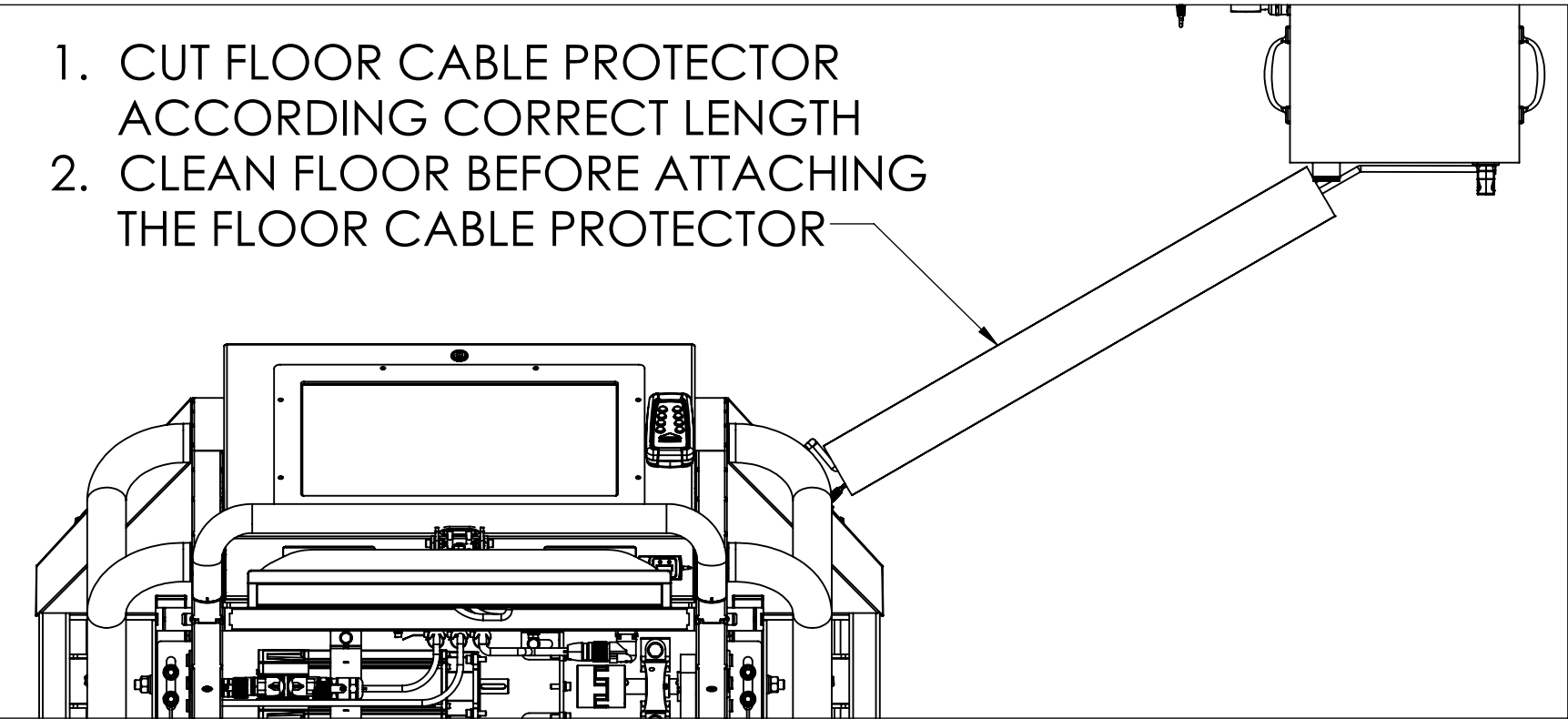

REPLACE POWER CABLE PLUG
WITH LOCAL STANDARD

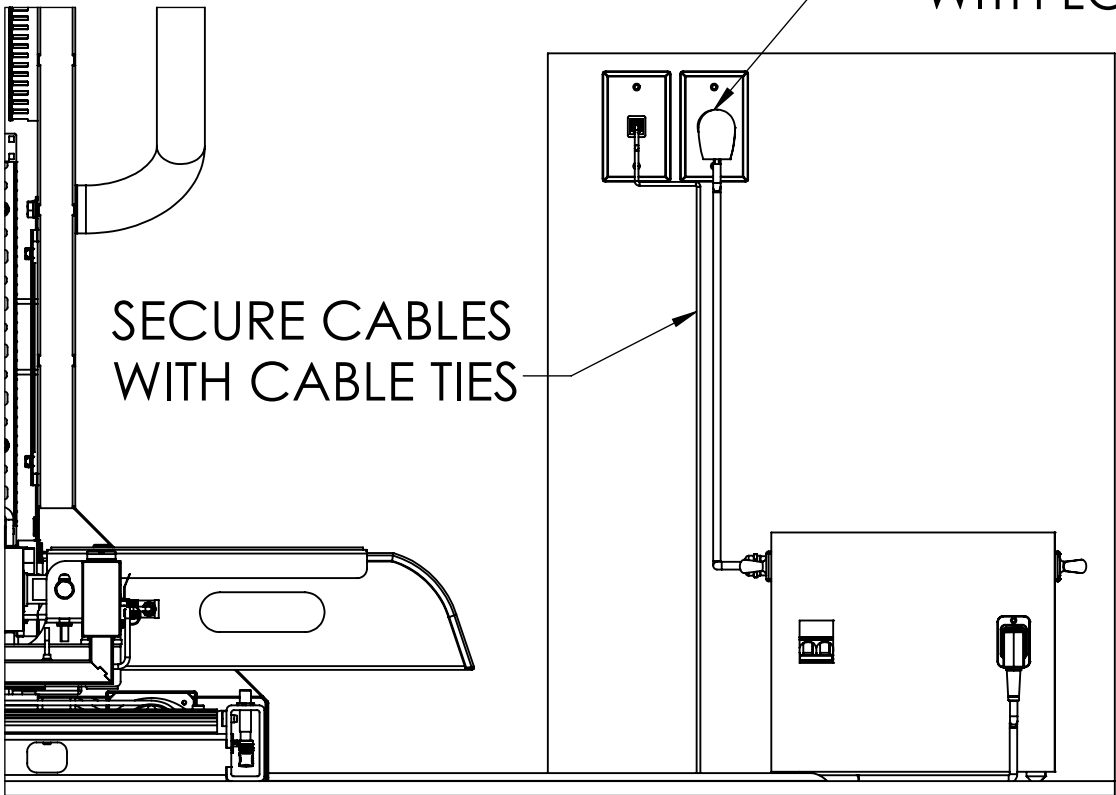

ADJUST POWER CABLE LENGTH
UP TO 3m BY ADJUSTING THE
PG13.5 METALIC NUT

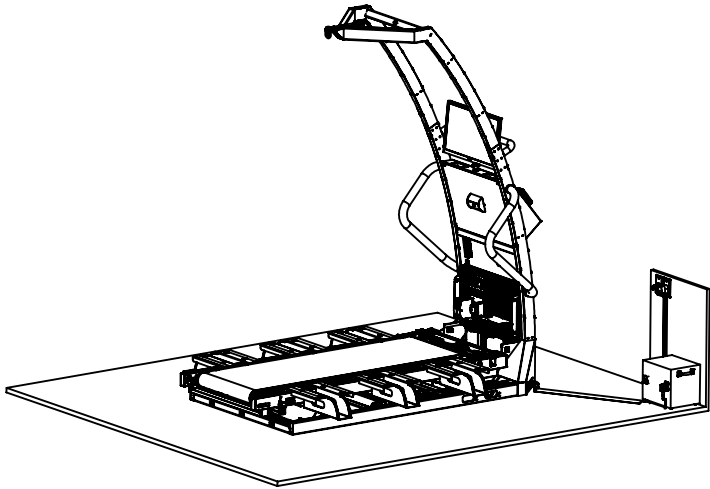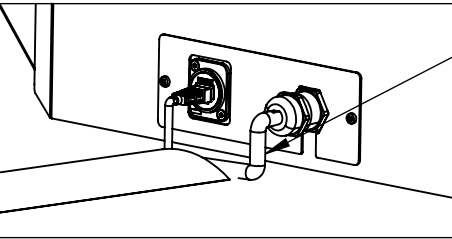

DOCUMENT NUMBER
DO-15-01-44_231122

DOCUMENT DESCRIPTION
Installation Guide

PROPRIETARY AND CONFIDENTIAL
THE INFORMATION CONTAINED IN THIS
DRAWING IS THE SOLE PROPERTY OF
MEDITOUCH LTD. ANY REPRODUCTION IN
PART OR AS A WHOLE WITHOUT THE
WRITTEN PERMISSION OF MEDITOUCH LTD.
IS PROHIBITED.

Assembly Drawing

THIRD
ANGLE
PROJECTION



ALL DIMENSIONS IN
mm



MediTouch

1 NIRIM STREET, TNUVOT, ISRAEL.

DRAWN BY
Ziv

SCALE
1:8

SHEET SIZE
A4

PAGE
16 of 26

TESTS & DOCUMENTATION

- 1. OPEN THE INSTALLATION REPORT FOR FILL IN
- 2. PERFORM ALL MECHANICAL AND ELECTRICAL TESTS LISTED IN SECTION 5 BEFORE CLOSING THE DEVICE WITH COVERS.
- 3. TAKE A FEW PICTURES OF THE INSIDE ACCORDING REQUIREMENTS IN THE SECTION 9
- 4. NOTE: THIS REPORT NEEDS TO BE SCANNED AND SENT TO MEDITOUCH FOR FUTURE WARRANTY PURPOSES

INSTALLATION REPORT
FIND INSIDE INSTALLATION FOLDER



DO-15-01-13



DOCUMENT NUMBER DO-15-01-44_231122	PROPRIETARY AND CONFIDENTIAL THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.	Assembly Drawing		 1 NIRIM STREET, TNUVOT, ISRAEL.	DRAWN BY Ziv	SHEET SIZE A4
DOCUMENT DESCRIPTION Installation Guide		THIRD ANGLE PROJECTION 	ALL DIMENSIONS IN mm	SCALE 1:8	PAGE 17 of 26	




FRONT COVER ASSEMBLY

F
E
D
C
B
A

F
E
D
C
B
A

ASSEMBLE FRONT COVER TWO HEX SCREWS

HEX ALLEN KEY
SIZE 4mm

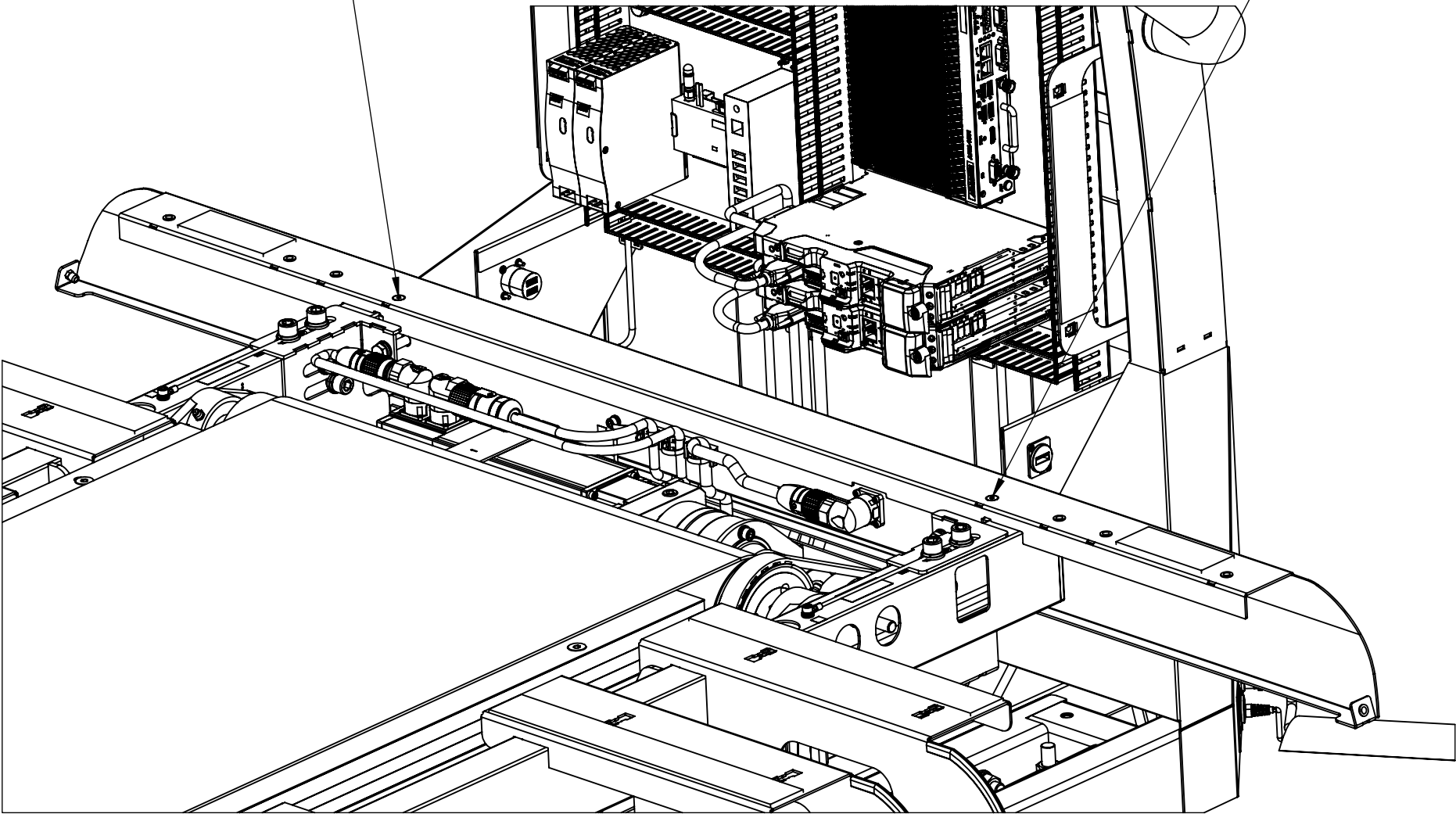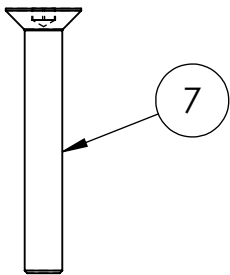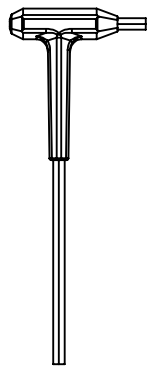

ITEM NO.	PART NUMBER	DESCRIPTION	QTY.
7	DS-02-01-44-D	Countersunk Hexagon Socket M5x35 DIN7991 SS	2




DOCUMENT NUMBER DO-15-01-44_231122	PROPRIETARY AND CONFIDENTIAL THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.	Assembly Drawing THIRD ANGLE PROJECTION 	ALL DIMENSIONS IN mm	 MediTouch 1 NIRIM STREET, TNUVOT, ISRAEL.	DRAWN BY Ziv	SHEET SIZE A4
DOCUMENT DESCRIPTION Installation Guide					SCALE 1:5	PAGE 18 of 26

A


A

8 7 6 5 4 3 2 1

BACK COVER ASSEMBLY

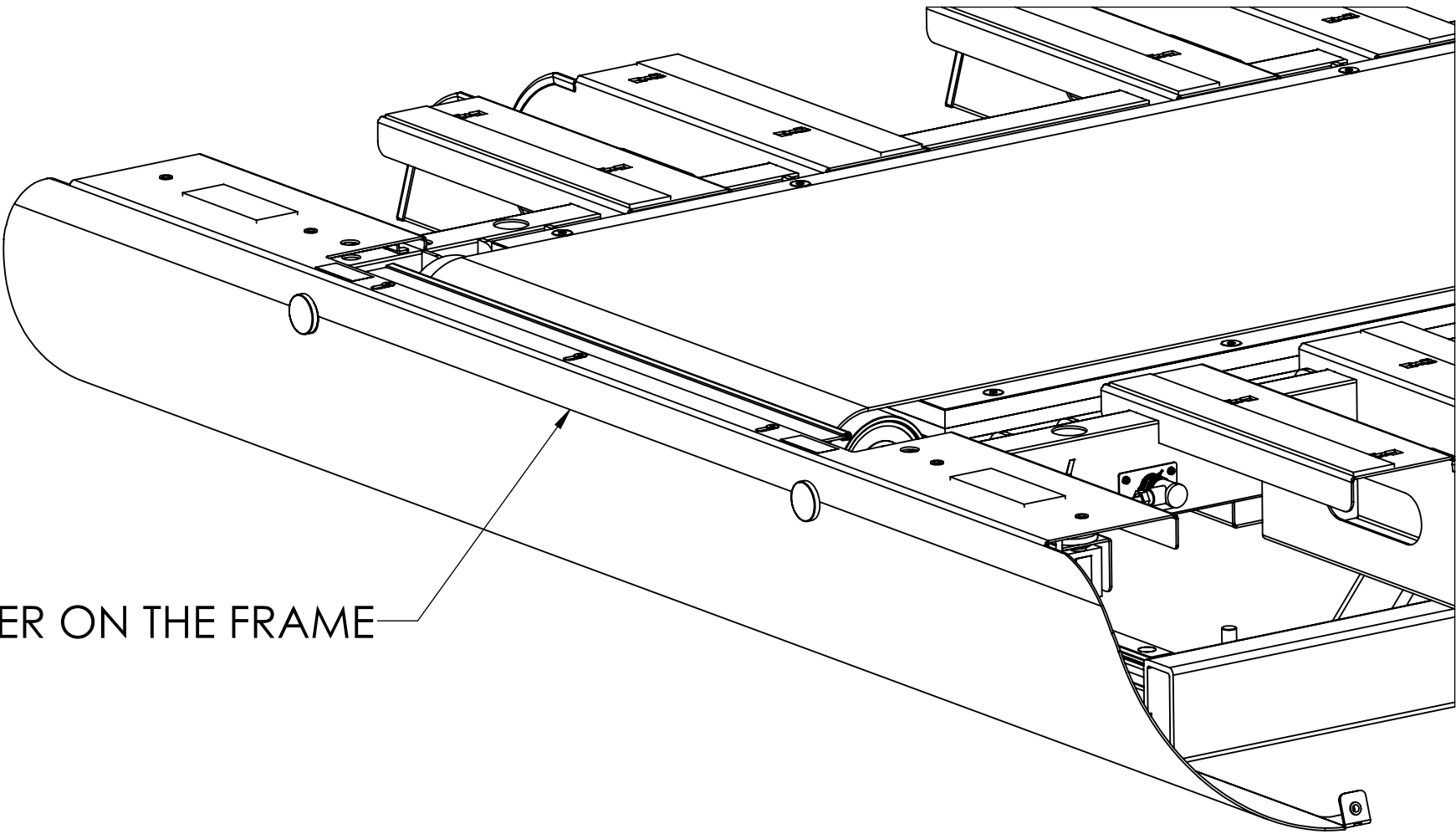

PLACE BACK COVER ON THE FRAME

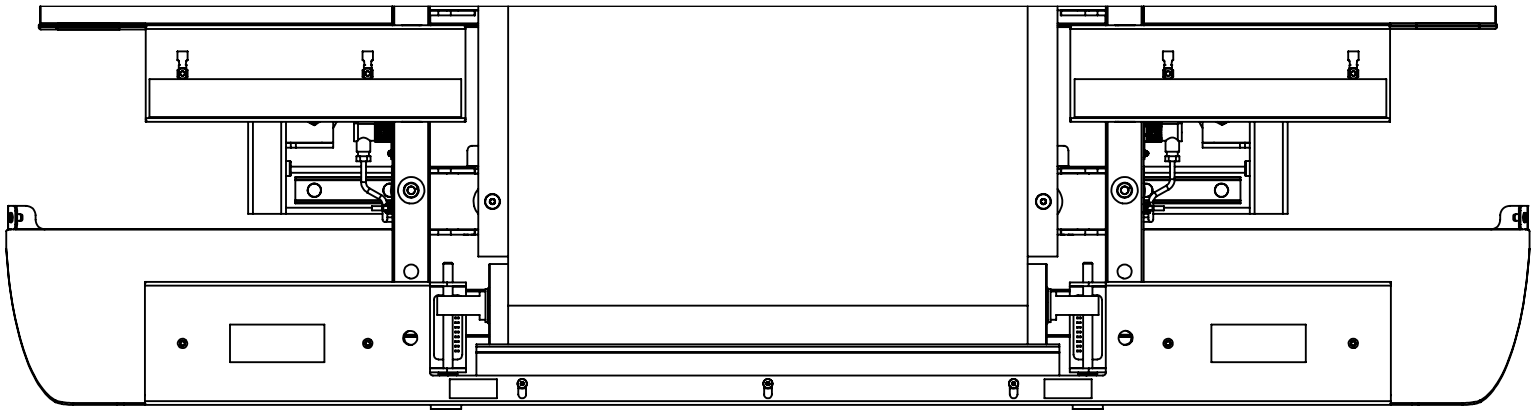

DOCUMENT NUMBER
DO-15-01-44_231122

DOCUMENT DESCRIPTION
Installation Guide

PROPRIETARY AND CONFIDENTIAL

THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.

Assembly Drawing

THIRD
ANGLE
PROJECTION



ALL DIMENSIONS IN
mm



1 NIRIM STREET, TNUVOT, ISRAEL.

DRAWN BY
Ziv

SCALE
1:5


SHEET SIZE
A4

PAGE
19 of 26

SIDE COVERS ASSEMBLY

FASTEN THE 3 SCREWS OF THE EDGE COVER
5mm FROM THE RUNNING BELT

HEX ALLEN KEY
SIZE 4mm

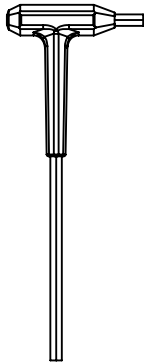

START ASSEMBLY
FROM HERE

CONTINUE
ASSEMBLY
FROM HERE

6



ITEM NO.	PART NUMBER	DESCRIPTION	QTY.
6	DS-02-01-44-A	Countersunk Hexagon Socket M5x12 DIN7991 SS	36



DOCUMENT NUMBER
DO-15-01-44_231122

DOCUMENT DESCRIPTION
Installation Guide

PROPRIETARY AND CONFIDENTIAL

THE INFORMATION CONTAINED IN THIS
DRAWING IS THE SOLE PROPERTY OF
MEDITOUCH LTD. ANY REPRODUCTION IN
PART OR AS A WHOLE WITHOUT THE
WRITTEN PERMISSION OF MEDITOUCH LTD.
IS PROHIBITED.

Assembly Drawing

THIRD
ANGLE
PROJECTION



ALL DIMENSIONS IN
mm



1 NIRIM STREET, TNUVOT, ISRAEL.

DRAWN BY
Ziv




SCALE
1:10

SHEET SIZE
A4


PAGE
20 of 26

GROUNDING SCREWS

HEX ALLEN KEY
SIZE 4mm

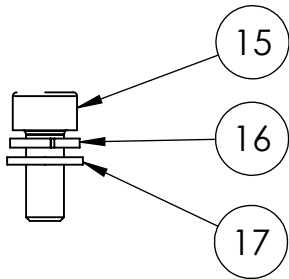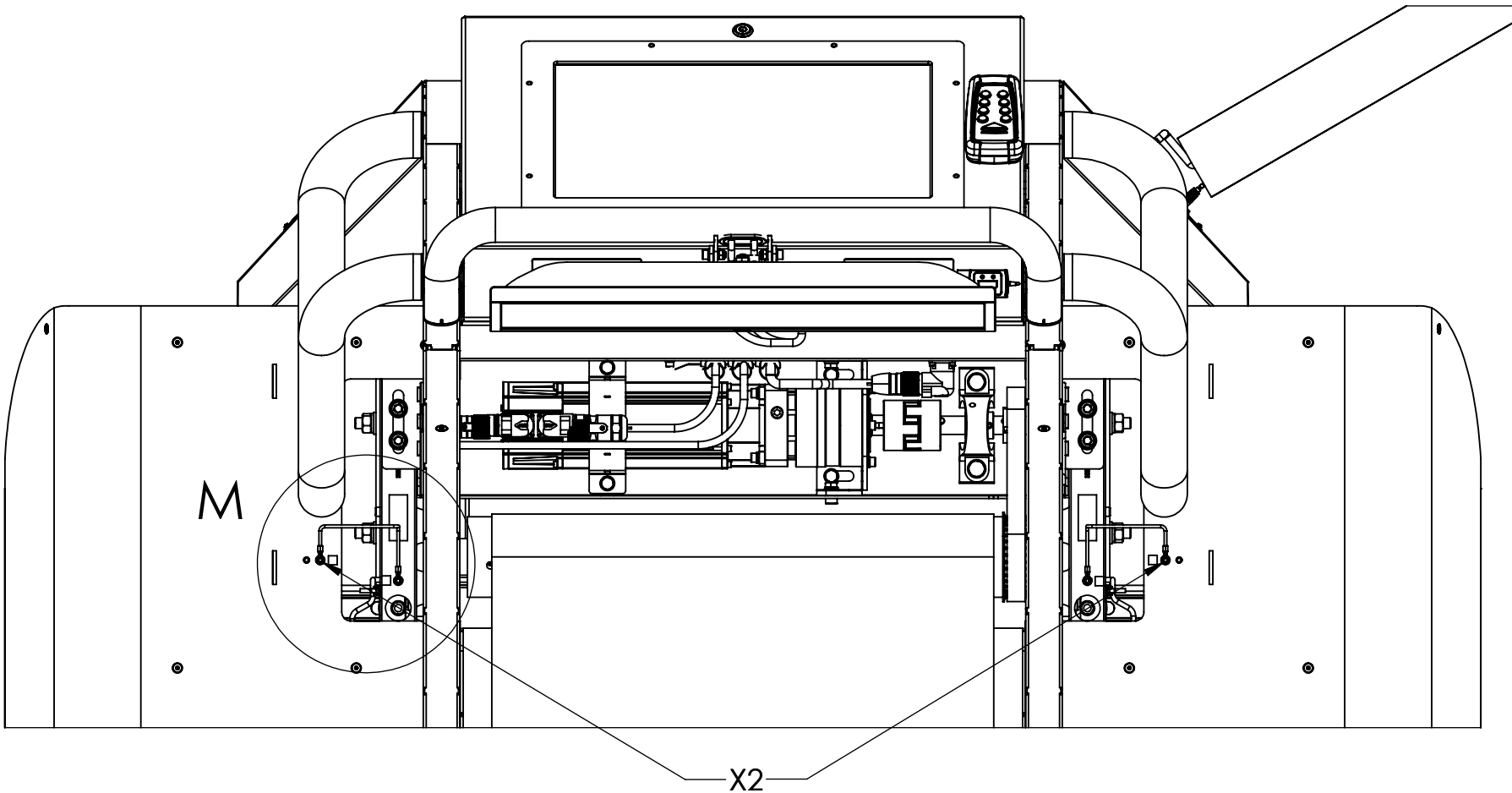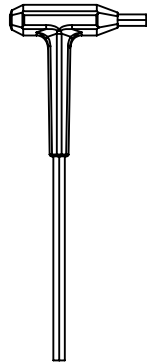

ITEM NO.	PART NUMBER	DESCRIPTION	QTY.
14	DS-02-01-24-A	Hexagon Socket M5X12 DIN912 Steel ZP 8.8	2
15	DS-02-01-30-F	Washer Spring Lock M5 DIN127-B Spring steel	2
16	DS-02-01-31-D	Washer M5 DIN125-1A Steel 140HV ZP	2

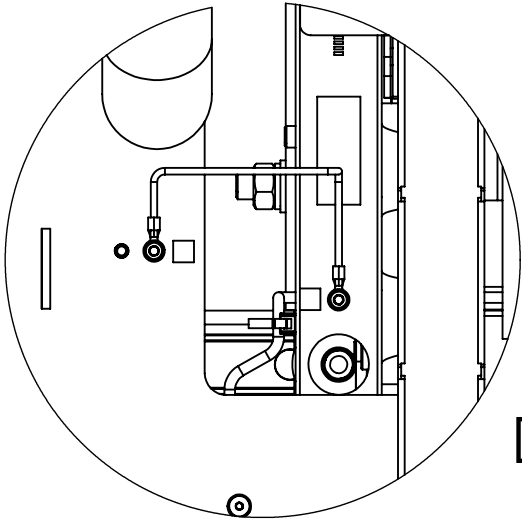




DETAIL M
SCALE 2 : 7



DOCUMENT NUMBER DO-15-01-44_231122	PROPRIETARY AND CONFIDENTIAL THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.	Assembly Drawing THIRD ANGLE PROJECTION	ALL DIMENSIONS IN mm	 MediTouch 1 NIRIM STREET, TNUVOT, ISRAEL.	DRAWN BY Ziv	SHEET SIZE A4
DOCUMENT DESCRIPTION Installation Guide					SCALE 1:7	PAGE 21 of 26

UPPER COVERS ASSEMBLY

HEX ALLEN KEY
SIZE 3mm

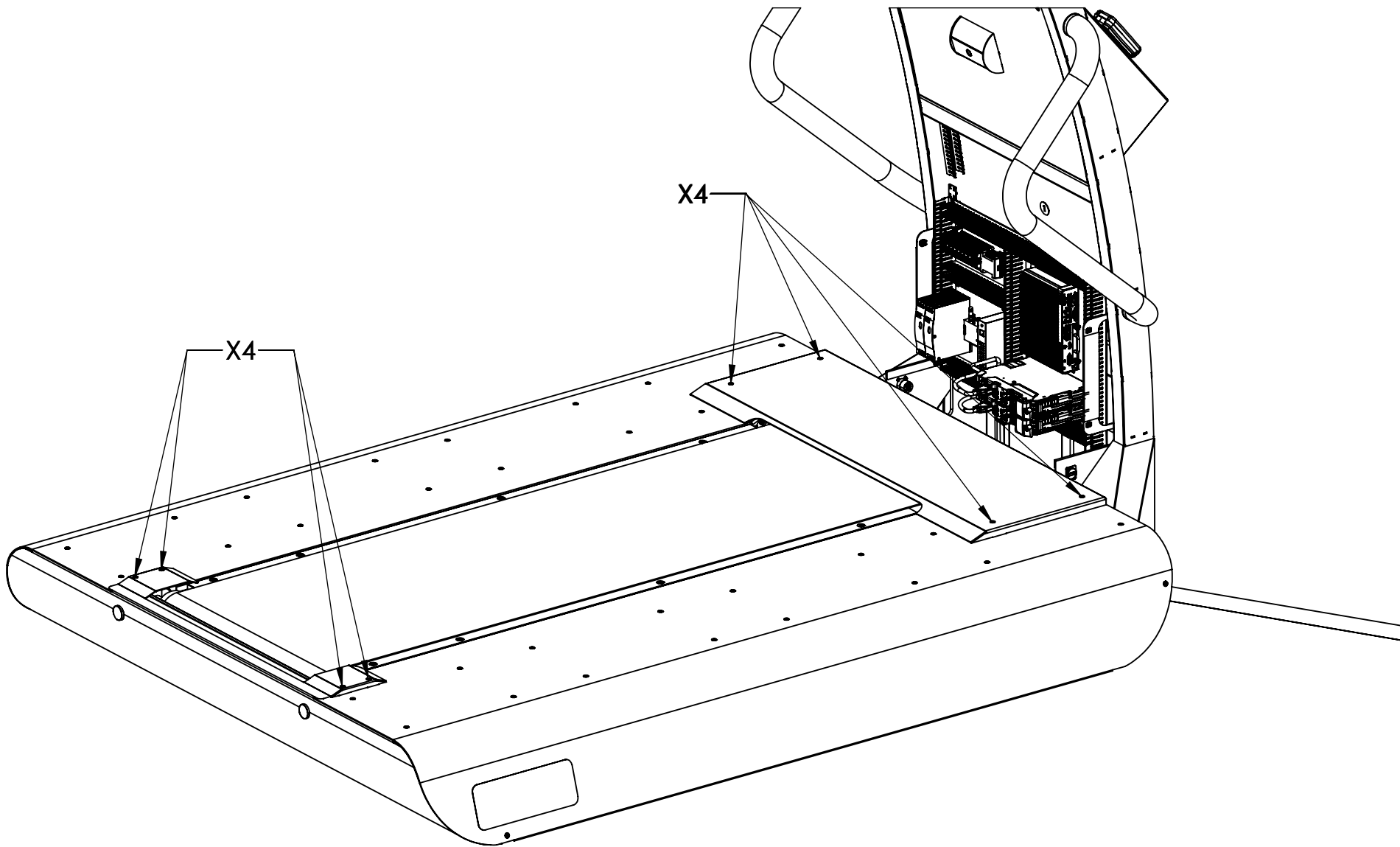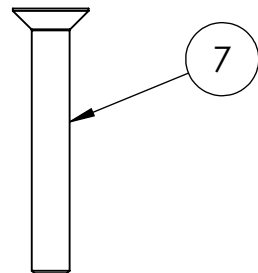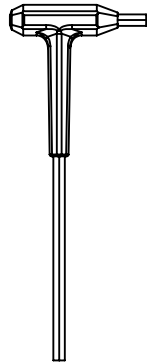


ITEM NO.	PART NUMBER	DESCRIPTION	QTY.
7	DS-02-01-44-D	Countersunk Hexagon Socket M5x35 DIN7991 SS	8





DOCUMENT NUMBER DO-15-01-44_231122	PROPRIETARY AND CONFIDENTIAL THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.	Assembly Drawing	 1 NIRIM STREET, TNUVOT, ISRAEL.	DRAWN BY Ziv	SHEET SIZE A4
DOCUMENT DESCRIPTION Installation Guide		THIRD ANGLE PROJECTION  ALL DIMENSIONS IN mm		SCALE 1:12	PAGE 22 of 26

POWER BOARD COVER ASSEMBLY

HEX ALLEN KEY
SIZE 3mm

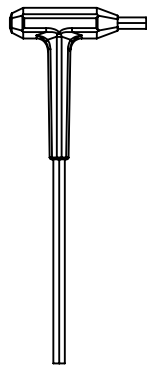

- 1. CONFIRM THIS POINT IS NOT TOUCHING THE FRONT UPPER COVER SURFACE
- 2. CONFIRM POWER COVER PARALEL TO THE FRONT UPPER COVER SURFACE

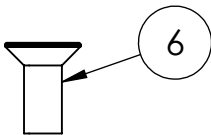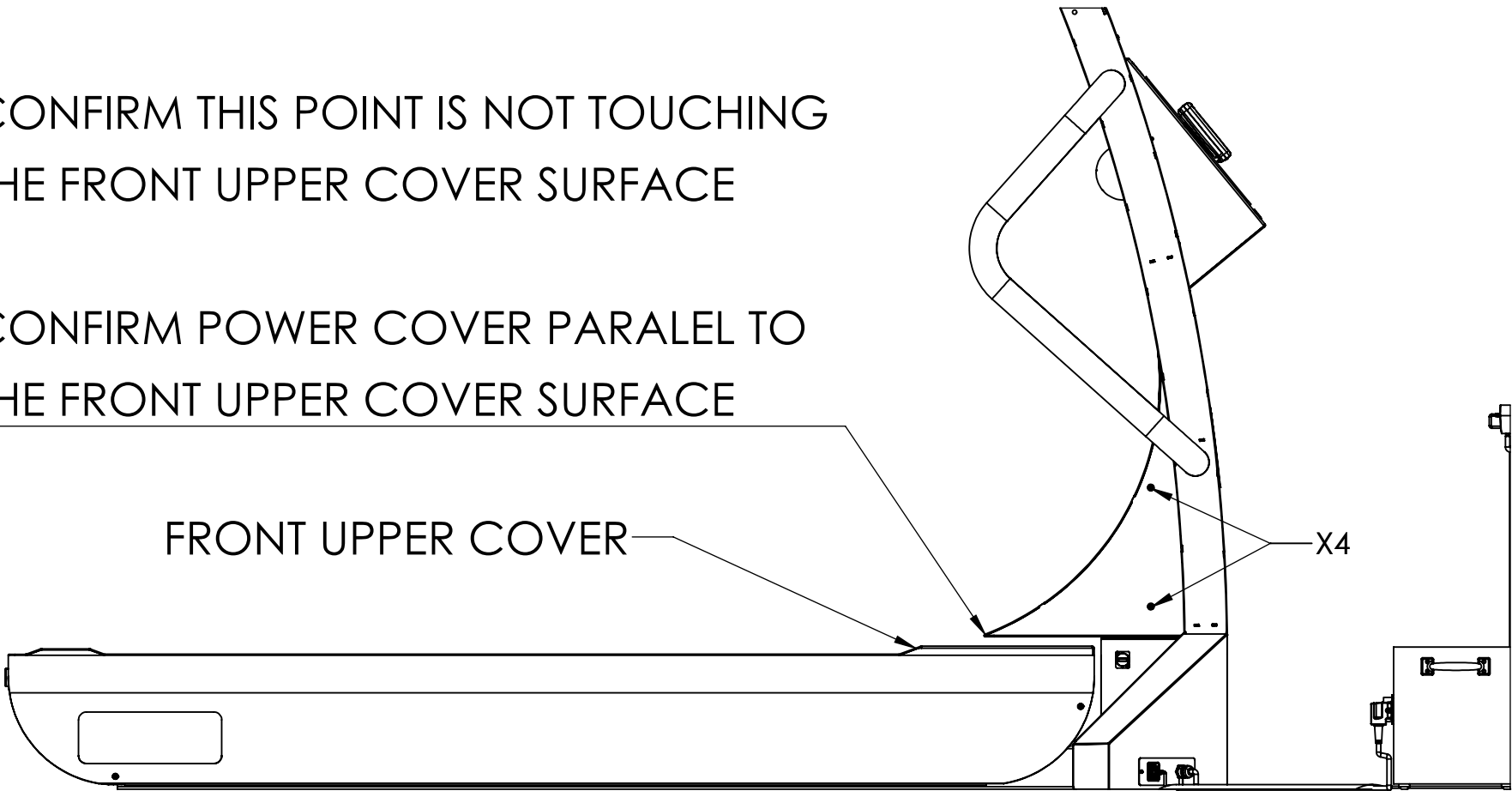

ITEM NO.	PART NUMBER	DESCRIPTION	QTY.
6	DS-02-01-44-A	Countersunk Hexagon Socket M5x12 DIN7991 SS	4




DOCUMENT NUMBER DO-15-01-44_231122	PROPRIETARY AND CONFIDENTIAL THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.	Assembly Drawing		ALL DIMENSIONS IN mm	 1 NIRIM STREET, TNUVOT, ISRAEL.	DRAWN BY Ziv	SHEET SIZE A4
DOCUMENT DESCRIPTION Installation Guide						SCALE 1:12	PAGE 23 of 26


KEEP OUT ZONE TAPE

- 1. CLEAN FLOOR WITH ALCOHOL BEFORE PLACING THE DUCT TAPE
- 2. PLACE DUCT TAPE ACCORDING DRAWING DIMENTIONS


2" YELLOW-BLACK
SAFETY DUCT TAPE

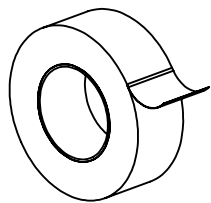

MEASURE TAPE

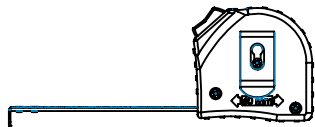


SNAP OFF
KNIFE

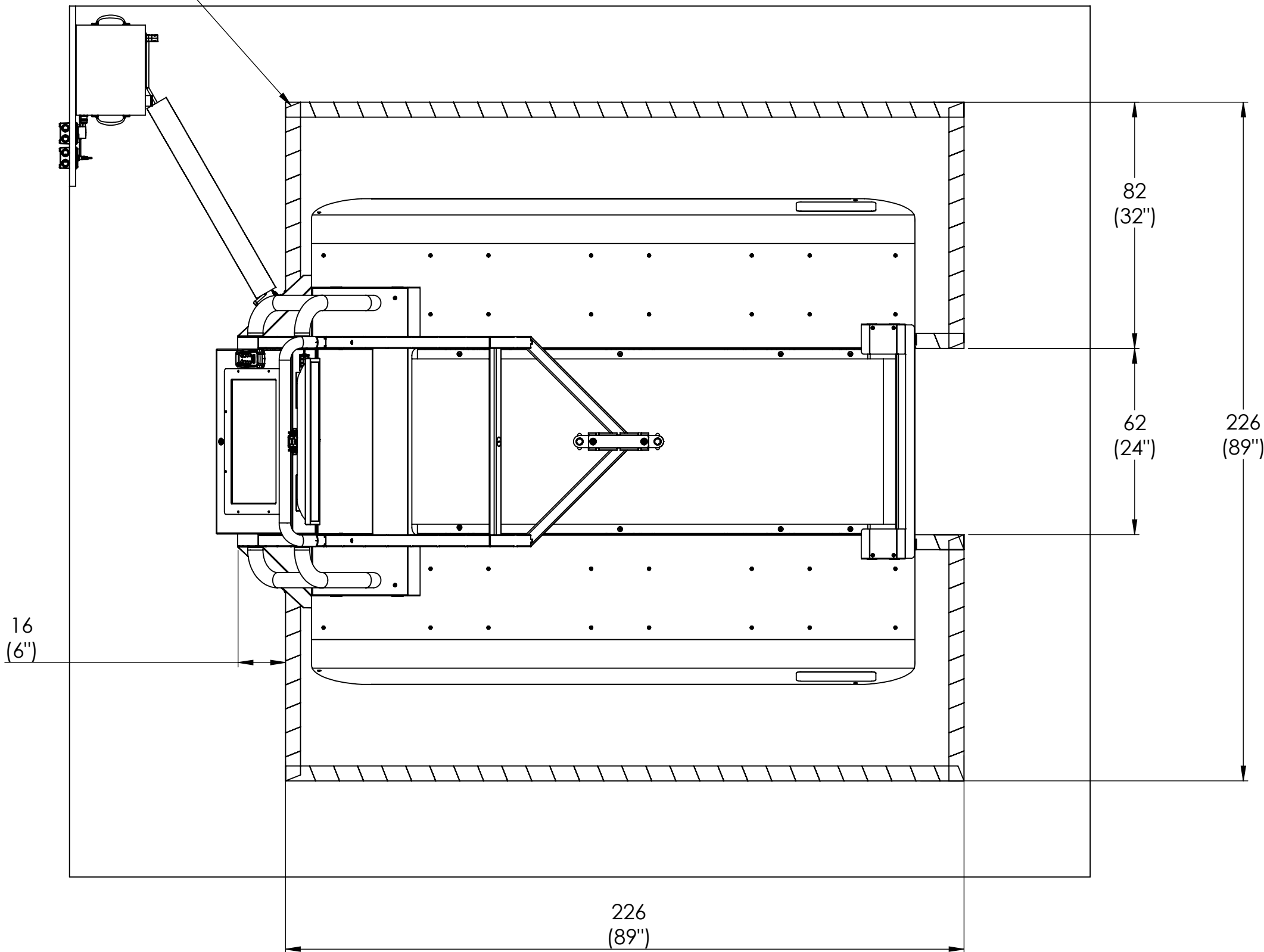

DOCUMENT NUMBER DO-15-01-44_231122	PROPRIETARY AND CONFIDENTIAL THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.	Assembly Drawing		 MediTouch 1 NIRIM STREET, TNUVOT, ISRAEL.	DRAWN BY Ziv	SHEET SIZE A4
DOCUMENT DESCRIPTION Installation Guide		THIRD ANGLE PROJECTION 	ALL DIMENSIONS IN mm		SCALE 1:15	PAGE 24 of 26

HANDHELD OPERATION

1. TURN ON THE BT AND LOGIN. ANY BUTTON PRESS WILL START COMMUNICATION OF THE HANDHELD AND THE RECEIVER
2. HANDHELD WILL TURN OFF IF NO BUTTON PRESSED DURING 30MIN
3. COMMUNICATION OF THE HANDHELD WILL INDICATED BY A GREEN BLINKING LED ON THE HANDHELD AND A GREEN HANDHELD ICON IN THE APPLICATION.
4. EMERGENCY BUTTON CAN BE ACTIVATED IN ALL APPLICATION STATES
5. ALL OTHER BUTTONS WILL BE OPERATIONAL DURING A SELECTED TREATMENT MODE ONLY
6. ENTER TO SETTINGS > THECNICAL > REMOTE AND SEE HANDHELD AND RECEIVER SYSTEM STATUS
7. HANDHELD AND RECEIVER ARE PAIRED AT MANUFACTURER AND COMMUNICATES ONLY ONE EACH OTHER

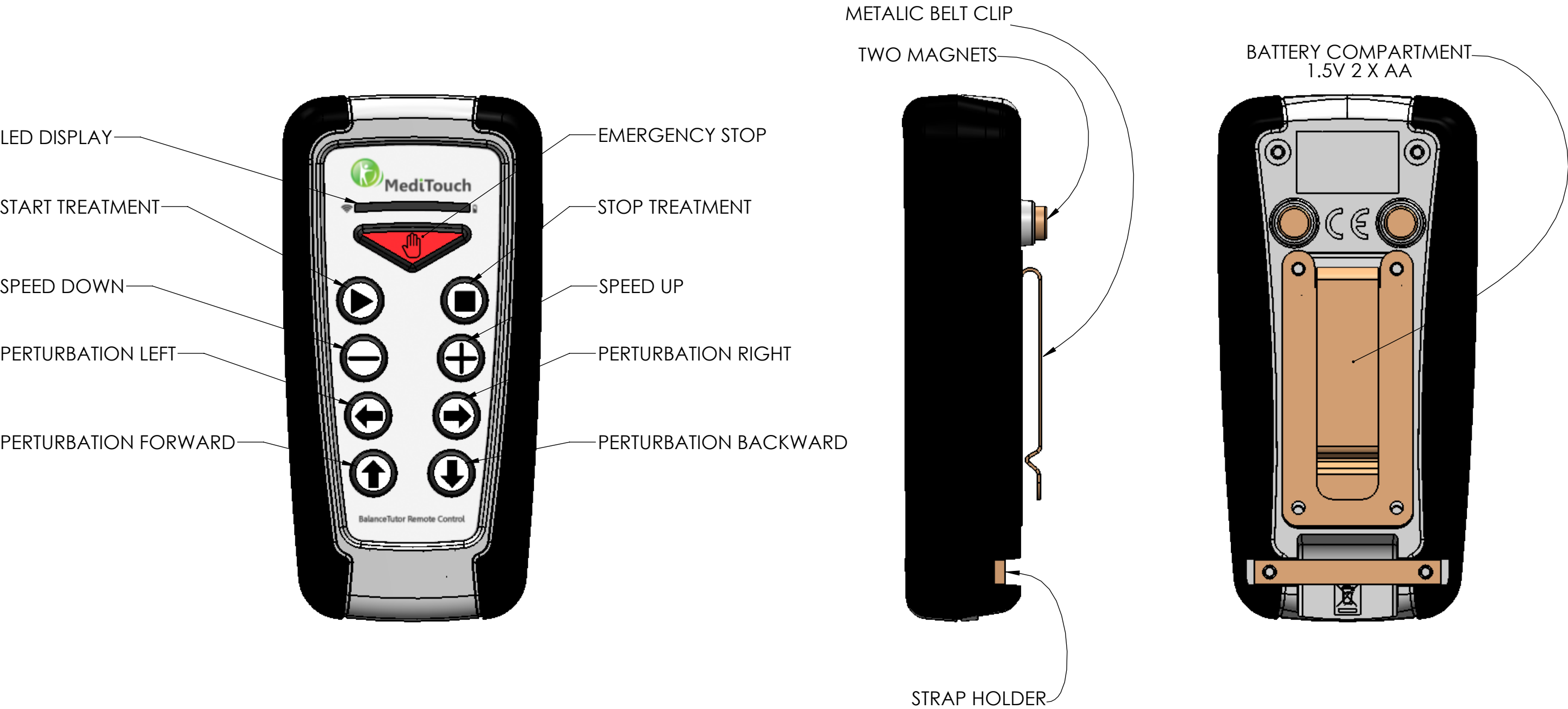

DOCUMENT NUMBER DO-15-01-44_231122	PROPRIETARY AND CONFIDENTIAL THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.	Assembly Drawing THIRD ANGLE PROJECTION	ALL DIMENSIONS IN mm	 MediTouch 1 NIRIM STREET, TNUVOT, ISRAEL.	DRAWN BY Ziv	SHEET SIZE A4
DOCUMENT DESCRIPTION Installation Guide					SCALE 1:12	PAGE 25 of 26

TESTS & DOCUMENTATION

- 1. FINALIZE ALL TESTS LISTED IN SECTION 5
- 2. FINALIZE ALL PICTURES REQUIRED IN SECTION 9
- 3. COMPLETE ALL OTHER SECTION
- 4. NOTE: THIS REPORT NEEDS TO BE SCANNED AND SENT TO MEDITOUCH FOR FUTURE WARRANTY PURPOSES

INSTALLATION REPORT
FIND INSIDE INSTALLATION FOLDER



DO-15-01-13



DOCUMENT NUMBER DO-15-01-44_231122	PROPRIETARY AND CONFIDENTIAL THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.	Assembly Drawing		 1 NIRIM STREET, TNUVOT, ISRAEL.	DRAWN BY Ziv	SHEET SIZE A4
DOCUMENT DESCRIPTION Installation Guide		THIRD ANGLE PROJECTION 	ALL DIMENSIONS IN mm	SCALE 1:8	PAGE 26 of 26	