




# BALANCETUTOR TREATMENT STATION

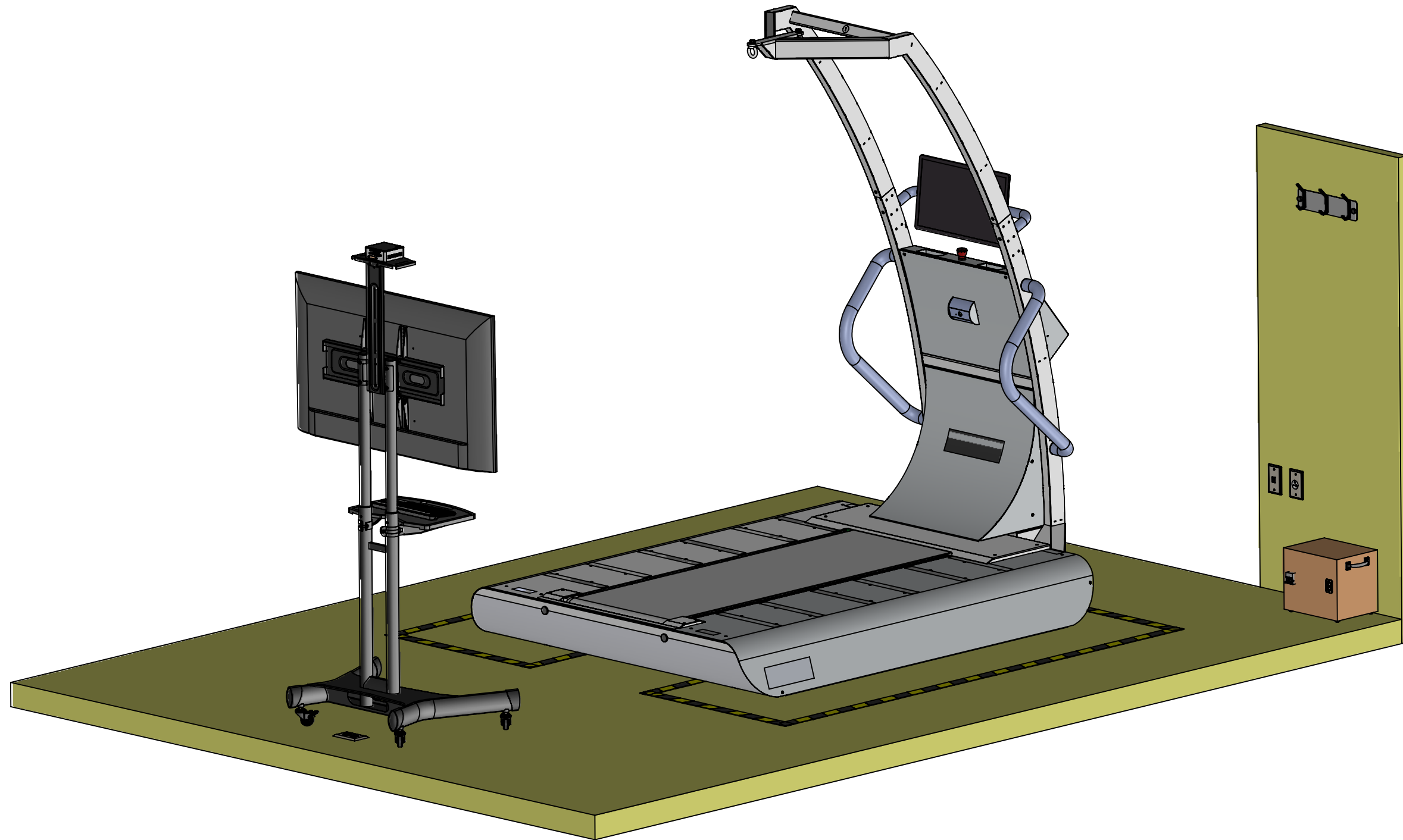


FOR US MARKET ONLY



DOCUMENT NUMBER <b>DO-15-01-43</b>	PROPRIETARY AND CONFIDENTIAL THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.	BalanceTutor Config 1	 THIRD ANGLE PROJECTION	ALL DIMENSIONS IN <b>inch</b>	 <b>MediTouch</b> 45 HAMELACHA STREET, NETANYA, ISRAEL.	DRAWN BY Ziv	SHEET SIZE A4
DOCUMENT DESCRIPTION BalanceTutor Recommended Configurations		SCALE 1:18				PAGE <b>1 of 9</b>	

# RECOMMENDED EQUIPMENT ARRANGEMENT

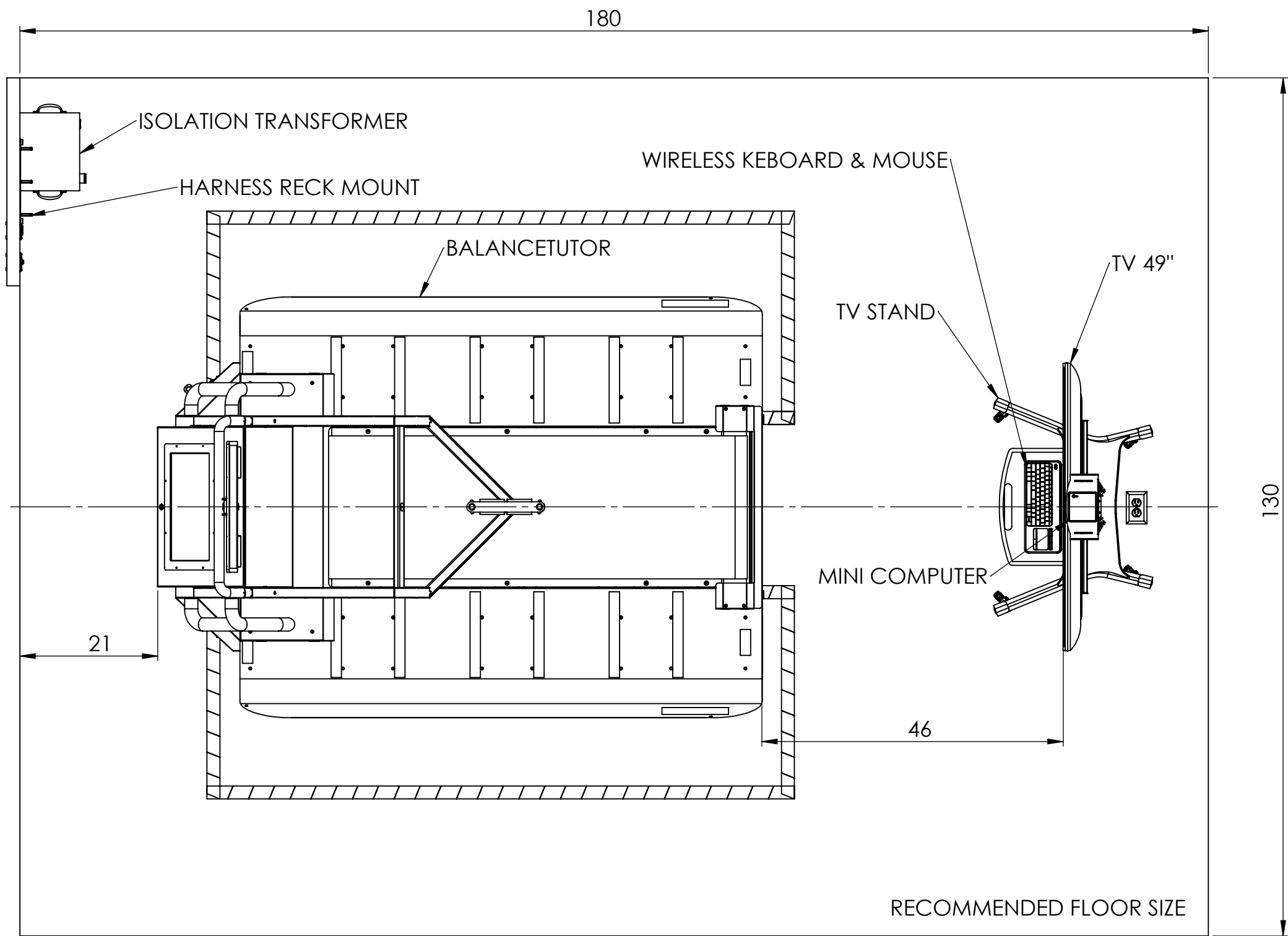
MINIMUM CEILING HEIGHT  
98 INCH



FOR US MARKET ONLY

DOCUMENT NUMBER <b>DO-15-01-43</b>	PROPRIETARY AND CONFIDENTIAL THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.	BalanceTutor Config 1	 <b>MediTouch</b> 45 HAMELACHA STREET, NETANYA, ISRAEL.	DRAWN BY Ziv	SHEET SIZE A4
DOCUMENT DESCRIPTION BalanceTutor Recommended Configurations	THIRD ANGLE PROJECTION 	ALL DIMENSIONS IN <b>inch</b>		SCALE 1:18	PAGE <b>2 of 9</b>


**SAFETY TAPE**

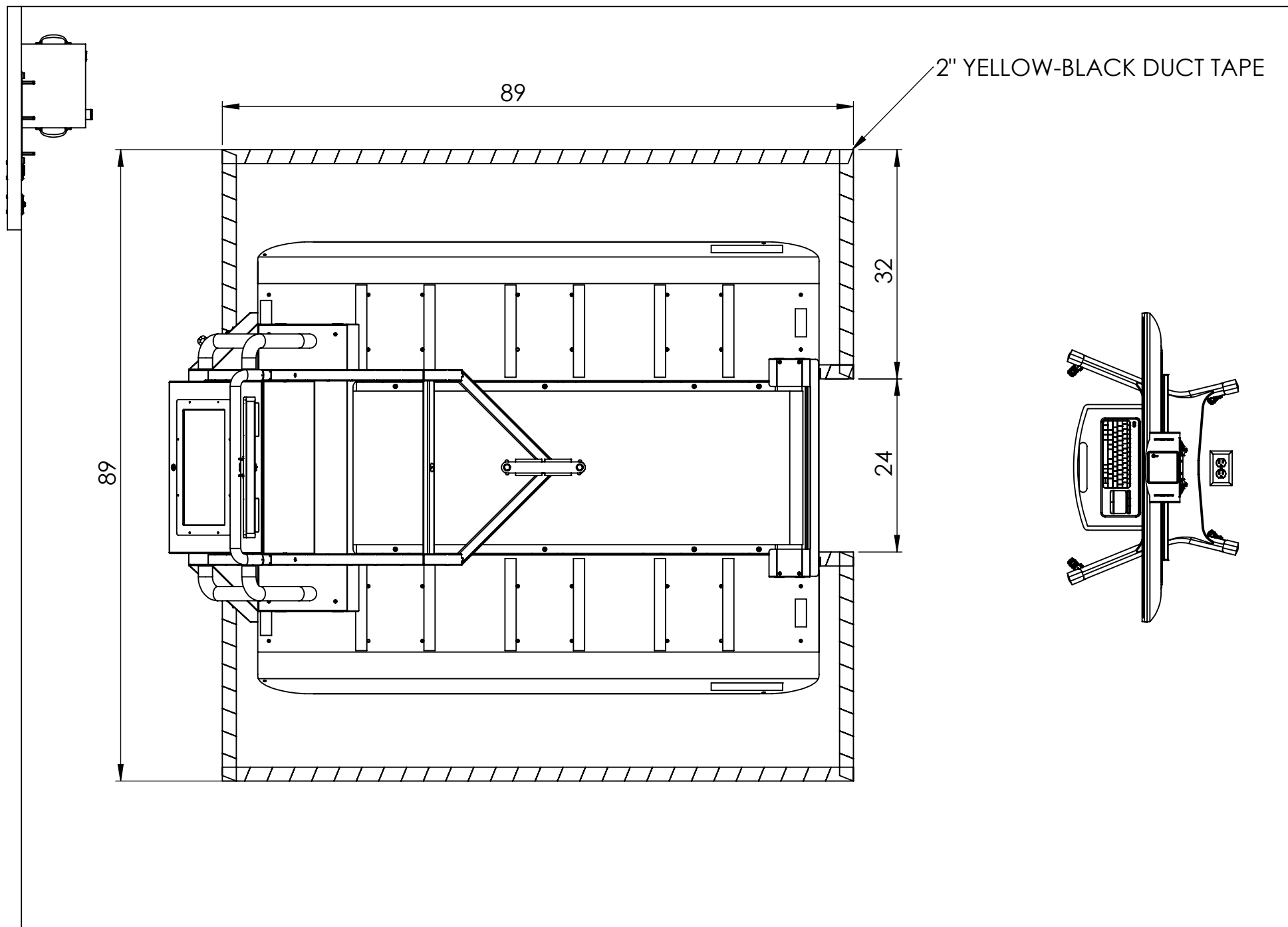


FOR US MARKET ONLY

DOCUMENT NUMBER <b>DO-15-01-43</b>	PROPRIETARY AND CONFIDENTIAL THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.	BalanceTutor Config 1	 ALL DIMENSIONS IN <b>inch</b>	 45 HAMELACHA STREET, NETANYA, ISRAEL.	DRAWN BY <b>Ziv</b>	SHEET SIZE <b>A4</b>
DOCUMENT DESCRIPTION BalanceTutor Recommended Configurations		THIRD ANGLE PROJECTION		SCALE <b>1:18</b>	PAGE <b>3 of 9</b>	

# TV STAND POWER OUTLET REQUIREMENTS

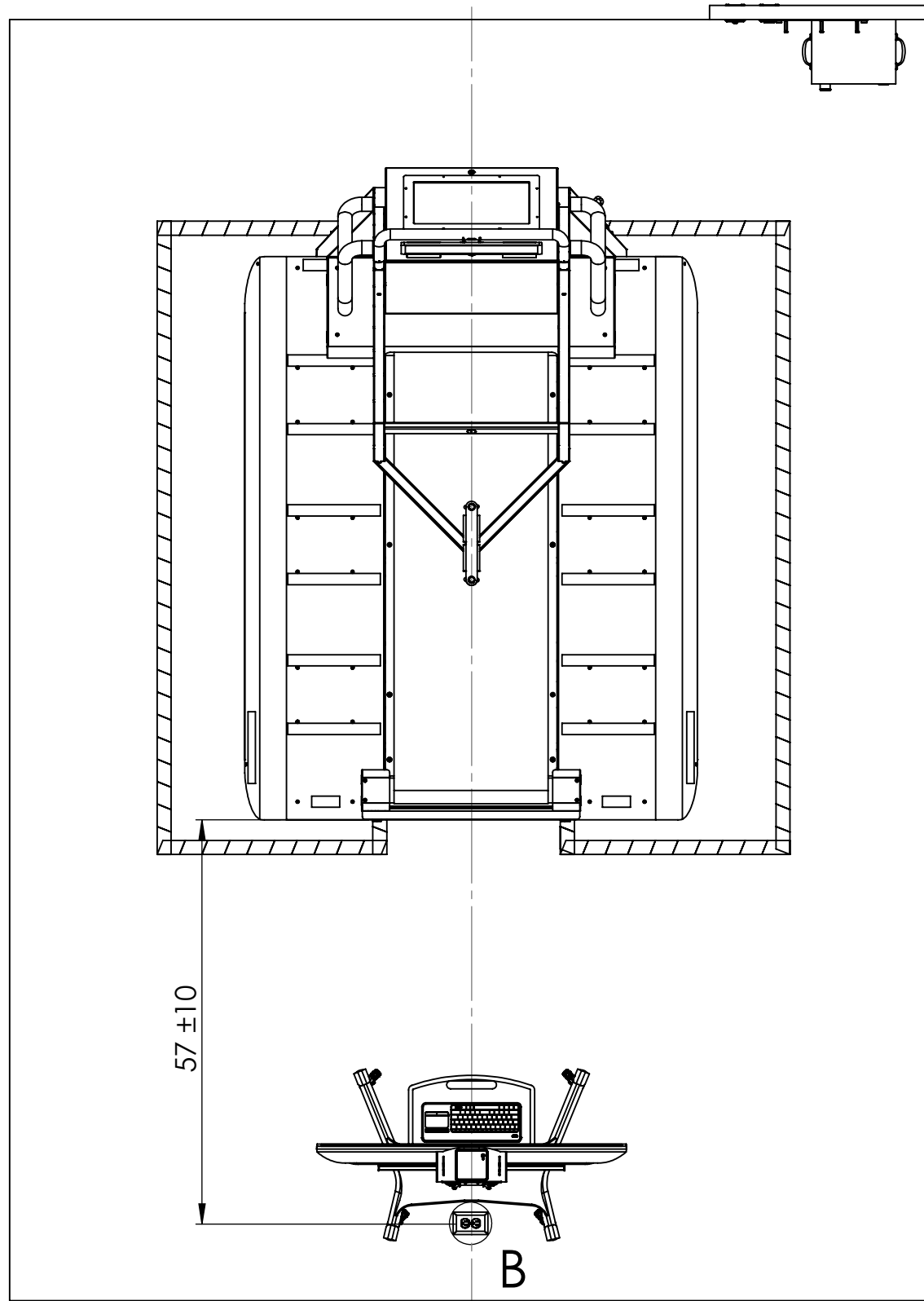
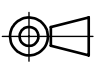


1. IN LINE FLOOR POWER BOX
2. TYPE: NEMA 5-15R
3. RATINGS: 120V (L-N-GND) 500VA
4. GRADE: IP68
5. PURPOSE: POWERING TV STAND

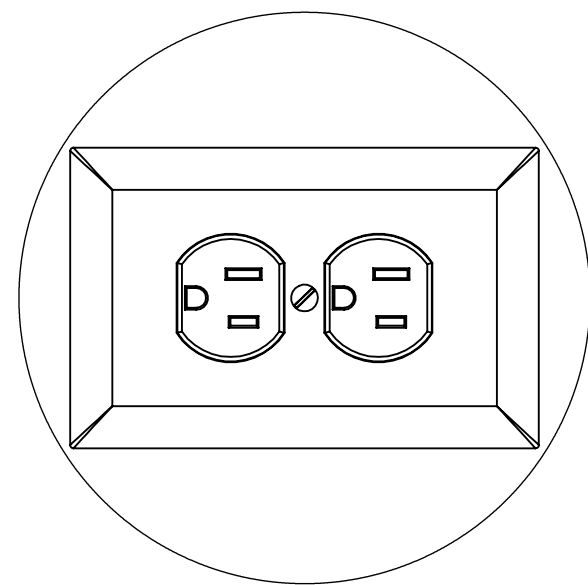



**DETAIL B**  
SCALE 1 : 2

FOR US MARKET ONLY

DOCUMENT NUMBER <b>DO-15-01-43</b>	PROPRIETARY AND CONFIDENTIAL THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.	BalanceTutor Config 1	 <b>MediTouch</b> 45 HAMELACHA STREET, NETANYA, ISRAEL.	DRAWN BY <b>Ziv</b>	SHEET SIZE <b>A4</b>
DOCUMENT DESCRIPTION BalanceTutor Recommended Configurations		THIRD ANGLE PROJECTION 	ALL DIMENSIONS IN <b>inch</b>	SCALE <b>1:18</b>	PAGE <b>4 of 9</b>

# BALANCETUTOR POWER OUTLET REQUIREMENTS

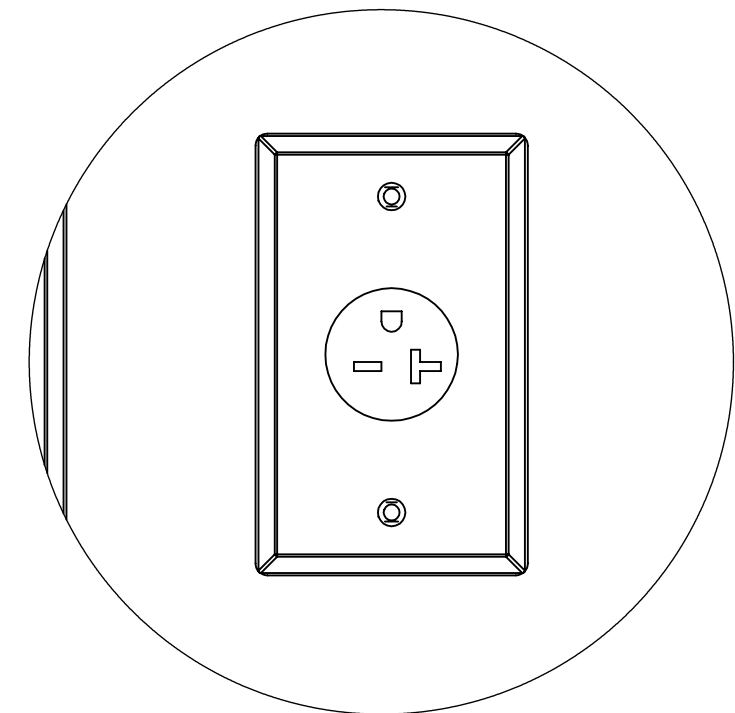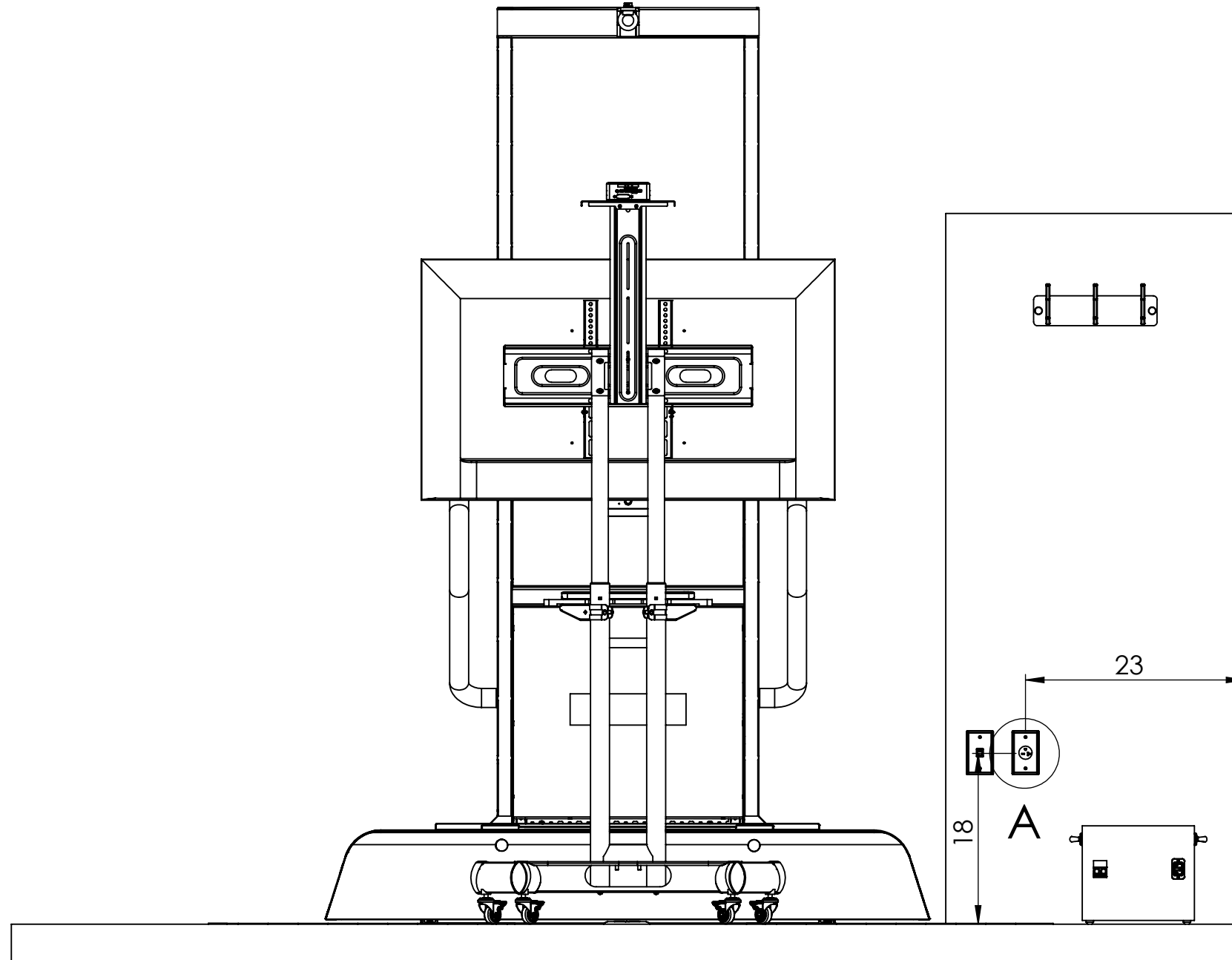

1. DEDICATED POWER LINE
2. OUTLET TYPE: NEMA 6-20R
3. RATINGS: 208V (L1-L2-GND) 2.3kVA
4. DUAL CIRCUIT BREAKER: 15AX2
5. PURPOSE: POWRING BALANCETUTOR ONLY



DETAIL A



SCALE 1:2

FOR US MARKET ONLY

DOCUMENT NUMBER DO-15-01-43	PROPRIETARY AND CONFIDENTIAL THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.	BalanceTutor Config 1		DRAWN BY Ziv	SHEET SIZE A4
DOCUMENT DESCRIPTION BalanceTutor Recommended Configurations		THIRD ANGLE PROJECTION 	ALL DIMENSIONS IN <b>inch</b>	SCALE 1:18	PAGE <b>5 of 9</b>
			45 HAMELACHA STREET, NETANYA, ISRAEL.		



# BALANCETUTOR REMOTE ACCESS REQUIREMENTS

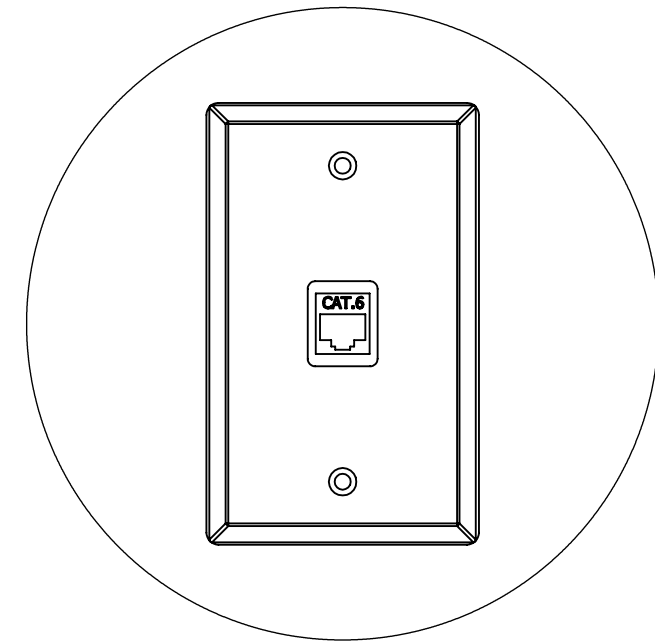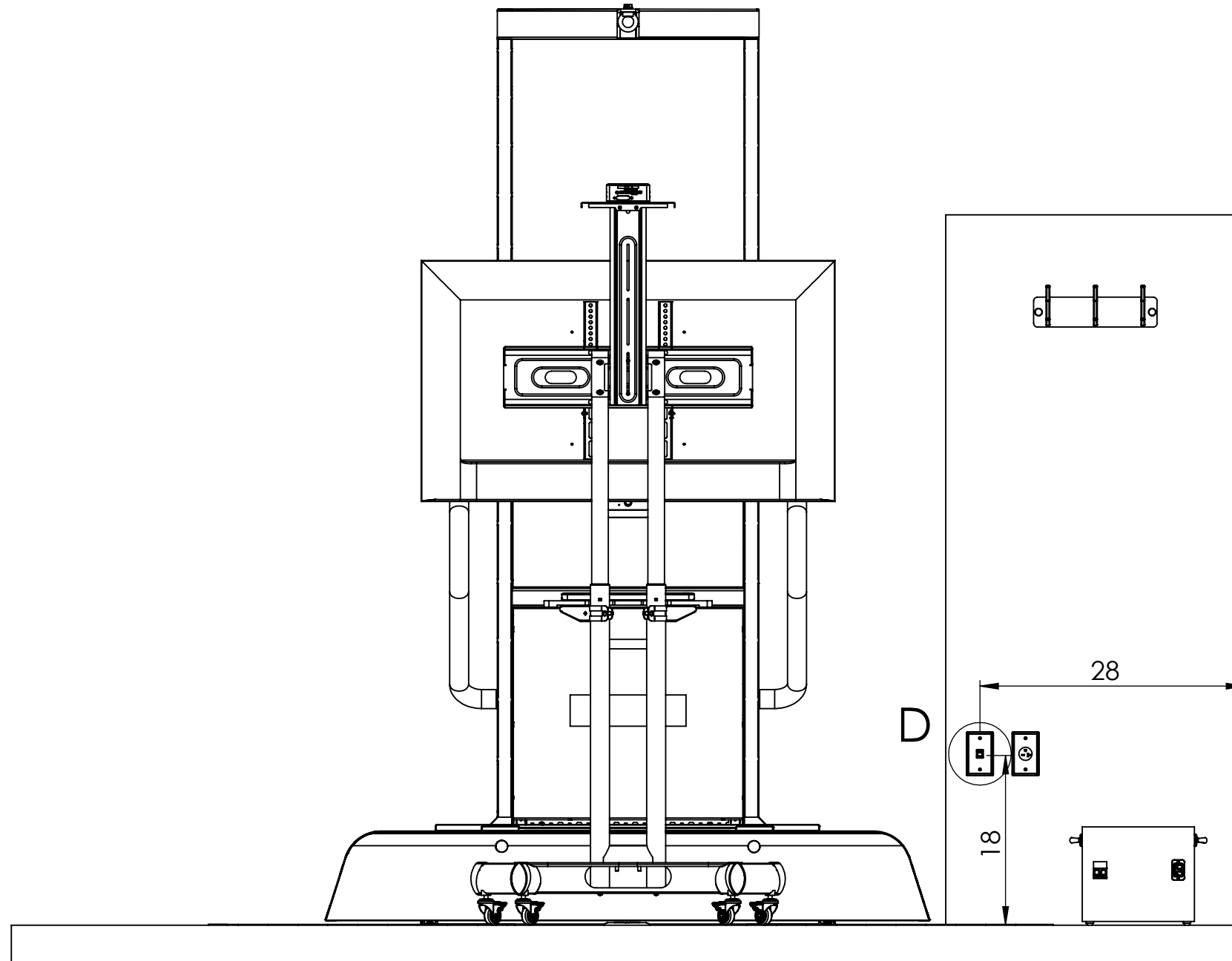1. CONNECTED TO WEB
2. TYPE: RJ45 CAT6
3. PURPOSE: REMOTE ACCES FOR SUPPORT & MAINTENANCE




**DETAIL D**  
SCALE 1 : 2


FOR US MARKET ONLY

DOCUMENT NUMBER <b>DO-15-01-43</b>	PROPRIETARY AND CONFIDENTIAL THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.	BalanceTutor Config 1	 THIRD ANGLE PROJECTION	 45 HAMELACHA STREET, NETANYA, ISRAEL.	DRAWN BY Ziv	SHEET SIZE A4
DOCUMENT DESCRIPTION BalanceTutor Recommended Configurations		ALL DIMENSIONS IN <b>inch</b>			SCALE 1:18	PAGE <b>6 of 9</b>

# HARNESS WALL RACK MOUNT INSTALLATION

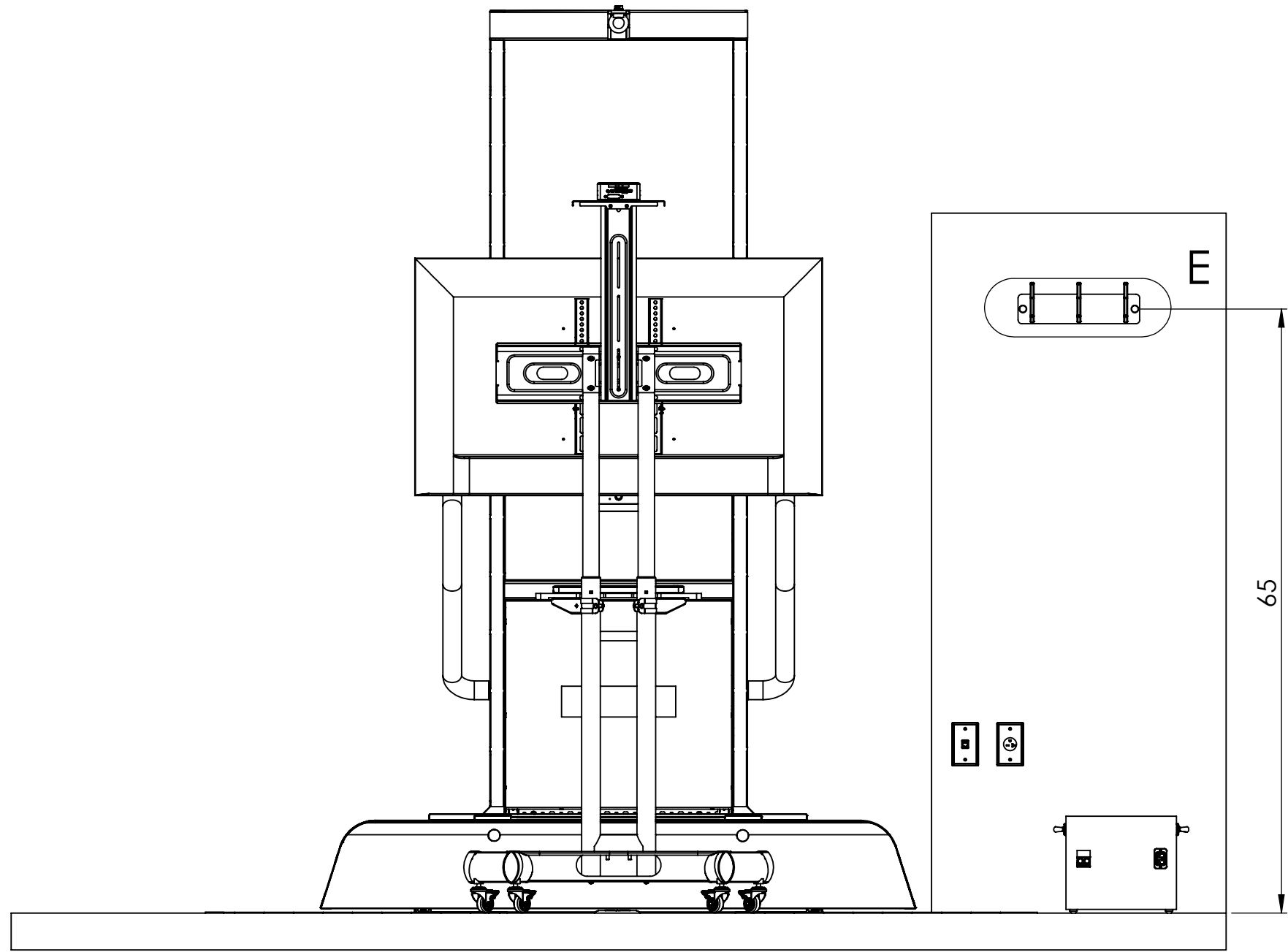

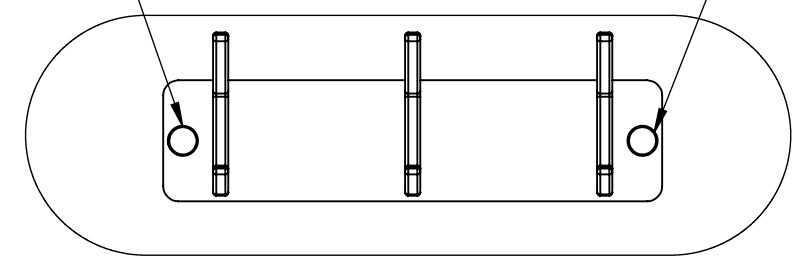

INSTALL THE HARNESS RACK WALL MOUNT USING TWO SCREWS SUPPLIED WITH THE ITEM





**DETAIL E**  
SCALE 1 : 5

FOR US MARKET ONLY

DOCUMENT NUMBER <b>DO-15-01-43</b>	PROPRIETARY AND CONFIDENTIAL THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.	BalanceTutor Config 1	 <b>MediTouch</b> 45 HAMELACHA STREET, NETANYA, ISRAEL.	DRAWN BY Ziv	SHEET SIZE A4
DOCUMENT DESCRIPTION BalanceTutor Recommended Configurations	THIRD ANGLE PROJECTION 	ALL DIMENSIONS IN <b>inch</b>		SCALE 1:18	PAGE <b>7 of 9</b>



# NETWORK ACCESS FOR THE 3DTUTOR SOFTWARE

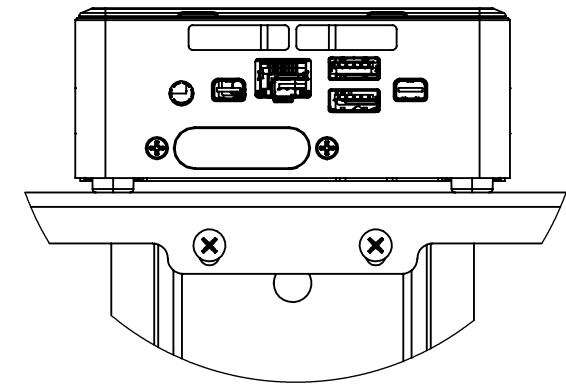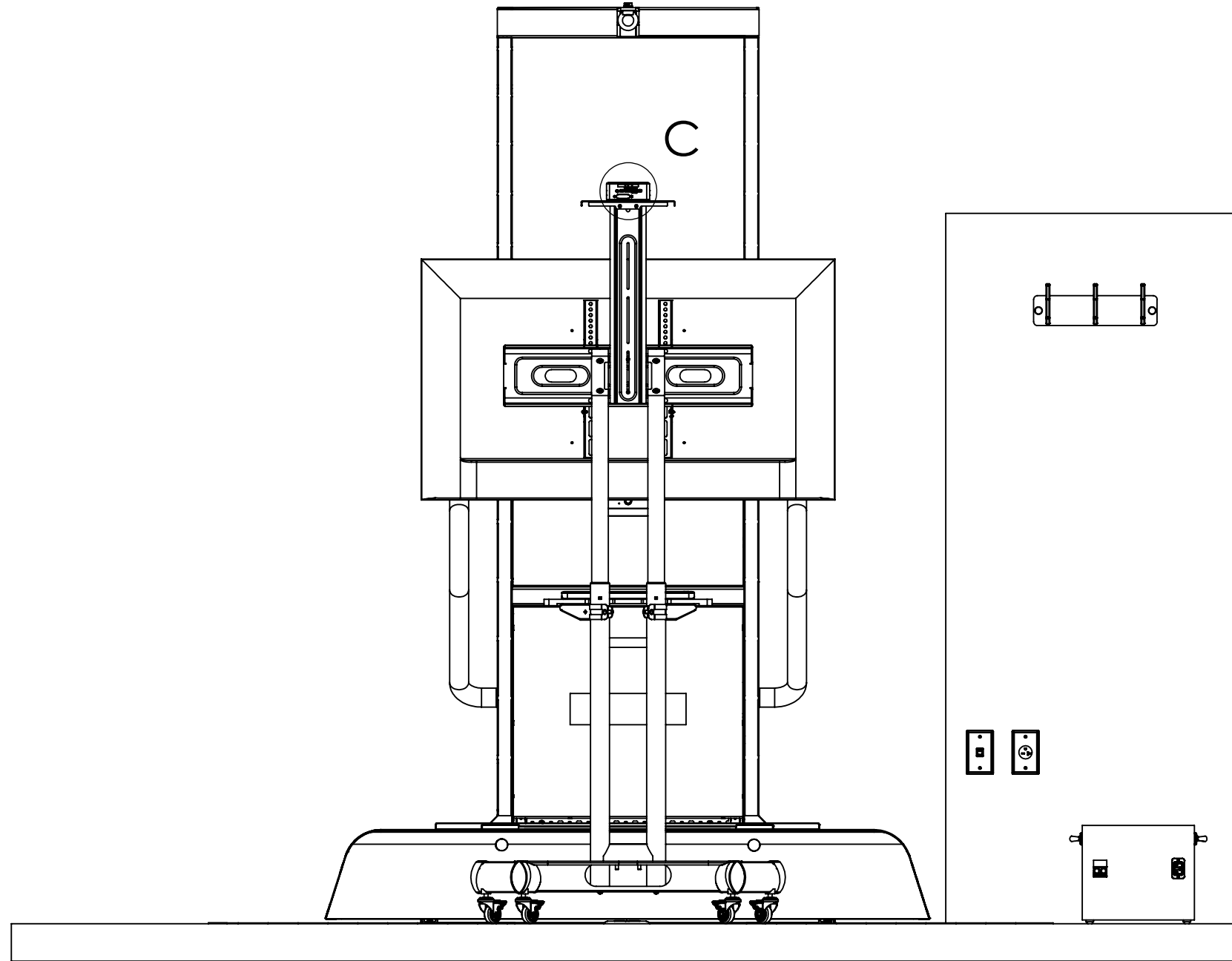1. CONSTANT NETWORK CONNECTION
2. ACCESS TO <https://meditutor.net> SITE
3. WIRELESS LAN CONNECTION






**DETAIL C**  
SCALE 1:2

FOR US MARKET ONLY



DOCUMENT NUMBER <b>DO-15-01-43</b>	PROPRIETARY AND CONFIDENTIAL THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.	BalanceTutor Config 1	 45 HAMELACHA STREET, NETANYA, ISRAEL.	DRAWN BY <b>Ziv</b>	SHEET SIZE <b>A4</b>
DOCUMENT DESCRIPTION BalanceTutor Recommended Configurations	THIRD ANGLE PROJECTION 	ALL DIMENSIONS IN <b>inch</b>		SCALE <b>1:18</b>	PAGE <b>8 of 9</b>

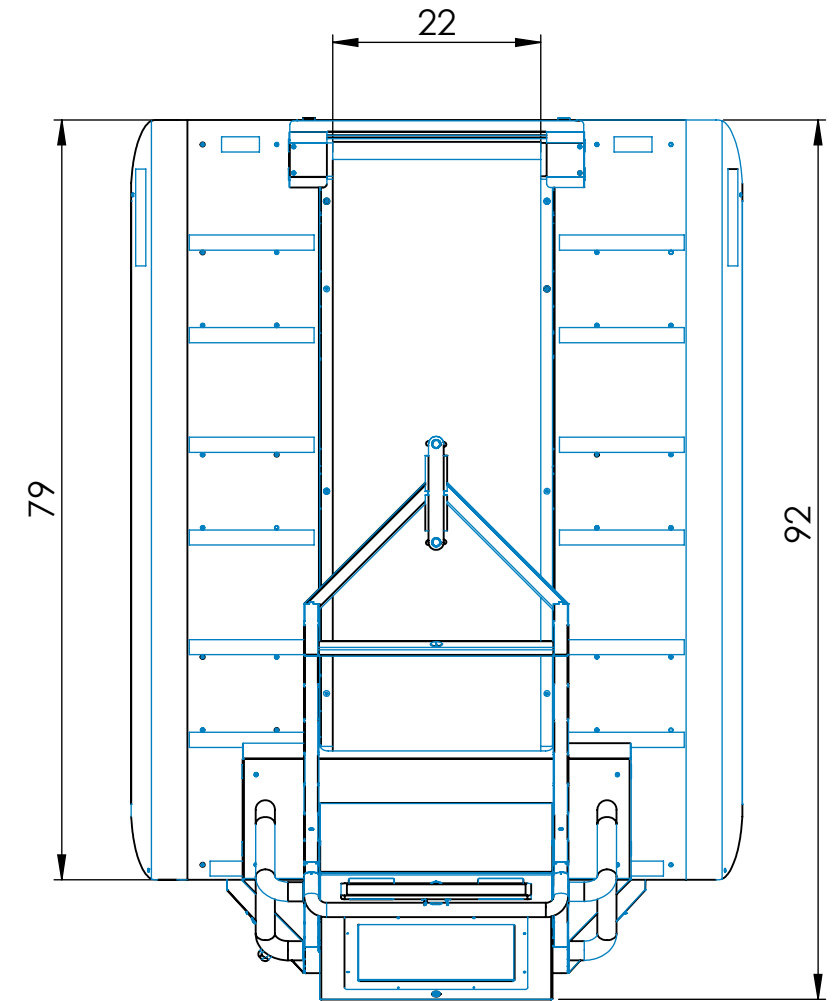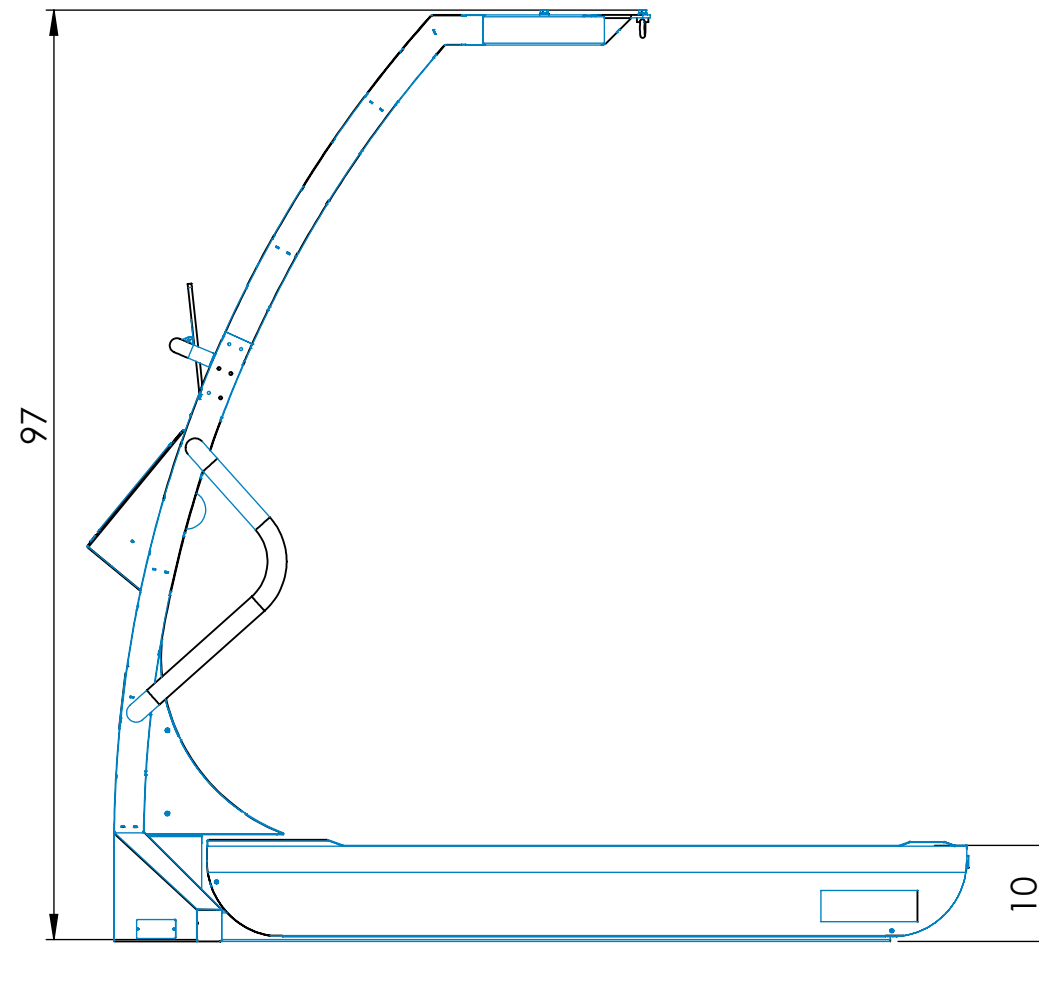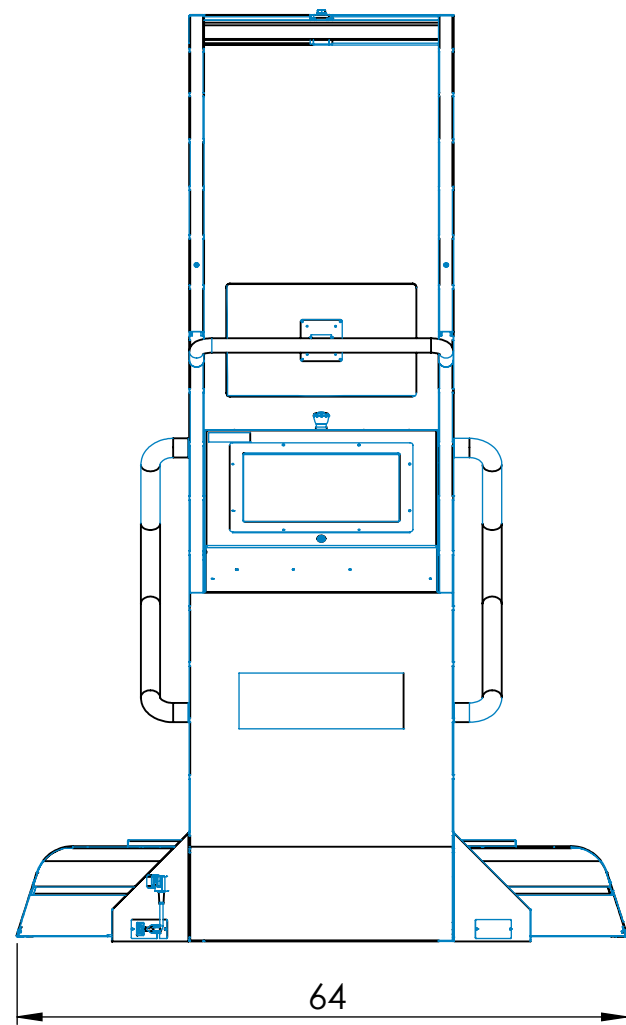

# GENERAL BALANCETUTOR DIMENTIONS



FOR US MARKET ONLY

DOCUMENT NUMBER <b>DO-15-01-43</b>	PROPRIETARY AND CONFIDENTIAL THE INFORMATION CONTAINED IN THIS DRAWING IS THE SOLE PROPERTY OF MEDITOUCH LTD. ANY REPRODUCTION IN PART OR AS A WHOLE WITHOUT THE WRITTEN PERMISSION OF MEDITOUCH LTD. IS PROHIBITED.	BalanceTutor Config 1	 THIRD ANGLE PROJECTION	ALL DIMENSIONS IN <b>inch</b>	 <b>MediTouch</b> 45 HAMELACHA STREET, NETANYA, ISRAEL.	DRAWN BY Ziv	SHEET SIZE A4
DOCUMENT DESCRIPTION BalanceTutor Recommended Configurations		SCALE 1:18				PAGE <b>9 of 9</b>	